
Pager Charger

Model BE1260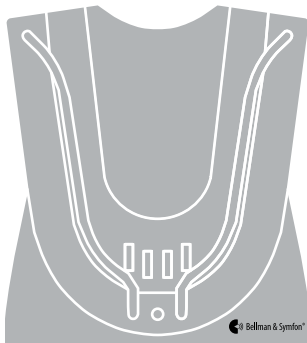

CZ

DE

DK

ES

FI

FR

GB

HU

IT

NL

NO

PL

PT

SE

SRB

Bellman Visit Charger

Postup instalace!

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Bellman Visit Charger

This is what to do!

1. Place the charger on a table or hang it on the wall. If it is placed vertically on a table the support legs should be fitted on the back of the charger.
2. Insert the mains adapter in a power point and connect one or two bed-shakers as required.
3. Insert Bellman Visit Pager in the charger. When the charging is completed (see section "Charging lamp") you can begin to use the pager.

Charging lamp

The charging lamp is green and placed on the front of the charger. It lights up when you insert the pager in the charger. This indicates that it is correctly connected and charging is in progress. When the battery is charged the charging lamp goes off. The charging lamp is off when the pager is not in the charger.

Back-up battery

The back-up battery is built into the charger and charges automatically when the charger's mains adapter is connected to a power point. In the event of a powercut the back-up battery supplies the bed-shakers with power so there is no need to worry about missing the alarm.

Bed-shaker

One or two bed-shakers can be connected to the charger in the vibrator outputs A and B. When the pager is in the charger it will not vibrate, the bed-shakers will vibrate instead according to the same pattern as the pager. The LEDs on the pager light up as usual.

Trouble shooting

A fault is not always serious. Follow the trouble shooting schedule below before calling for service.

Symptom	Procedure
The pager does not charge when it is in the charger.	Check that the pager is placed correctly in the charger. When the pager is inserted in the charger the charging lamp lights up.
The bed-shakers do not vibrate when the alarm is set.	Check that the bed-shakers are correctly connected to the charger and that the pager is correctly inserted in the charger.

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