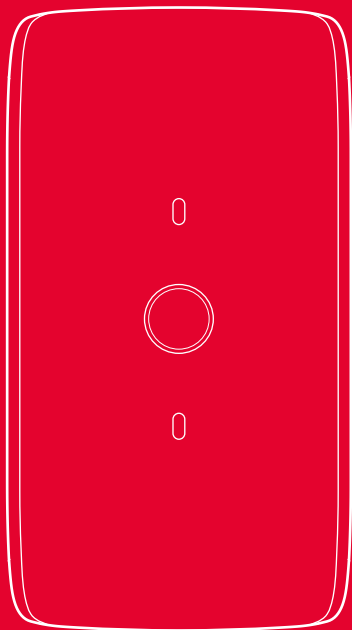
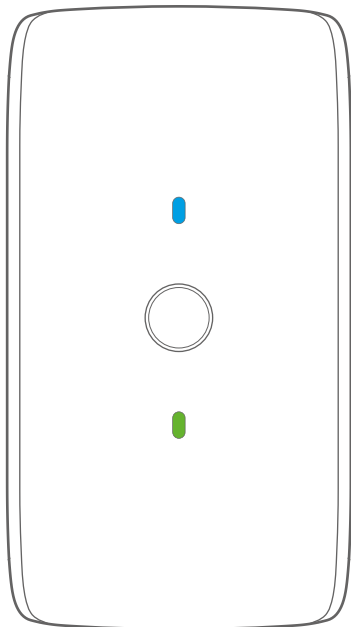
BE2022
Maxi Pro TV Streamer

BE2022 Maxi Pro TV Streamer

4	User manual
16	Bedienungsanleitung
28	Brugervejledning
40	Manual del usuario
52	Käyttöopas
64	Mode d'emploi
76	Manuale d'uso
88	Gebruikshandleiding
100	Brukermanual
112	Instrukcja obsługi
124	Bruksanvisning
136	Používateľská príručka
148	用戶手冊
160	用戶手冊
172	ユーザーマニュアル

EN
DE
DK
ES
FI
FR
IT
NL
NO
PL
SE
SK
CN _{SI}
CN _{TR}
JP

Read this first

This booklet guides you on how to use and maintain your new Maxi Pro TV Streamer. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

About the TV streamer

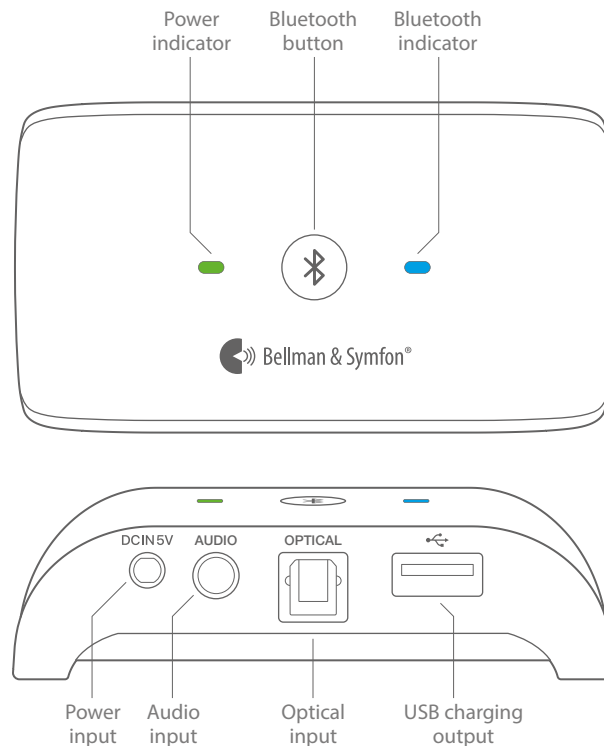
The Maxi Pro TV Streamer enables you to hear the TV at a volume that suits you, without affecting the volume for anyone else in the room. It streams the sound in full stereo via Bluetooth to your Maxi Pro Personal Amplifier.

Intended user group People of all ages.

Intended user A user that wants to control tuning and listening volume.

Principle of operation The audio product family consists of several amplifiers and sound transmitters which have been specifically developed to provide sound enhancement even in demanding situations. Depending on the assigned function of the specific amplifier or sound transmitter, different microphones can be used to pick up direct sound or enhance the ambient sound.

Overview

EN

What's in the box

Along with the TV streamer, the following items are included:

A mini-stereo cable

A mini-stereo-to-RCA adapter

A fiber optic cable

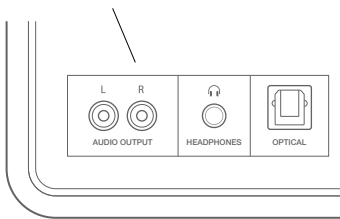
A DC-connector-to-USB cord

What you need to get started

To set up your TV streamer, you also need:

A TV with an RCA, headphone or optical audio port

A Maxi Pro Personal Amplifier

Setting up the TV streamer

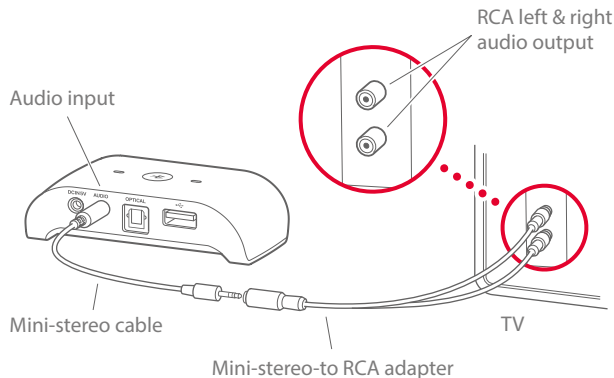
Step 1: Connecting audio

There are several ways to connect the TV streamer to your TV. It depends on the type of audio ports your TV is equipped with.

Using the TV's RCA output

- 1 Connect one end of the mini-stereo cable to the audio input on the back of the TV streamer.
- 2 Connect the other end to the mini-stereo-to-RCA adapter.
- 3 Connect the RCA plugs to the audio output on your TV.


You can also connect the mini-stereo cable directly to the headphone output on your TV. Please note that this may mute the speakers on some TV models.

Using the TV's optical output

If your TV features an optical output, you can use the fiber optic cable to connect the TV streamer to your TV. Here is how you connect it:

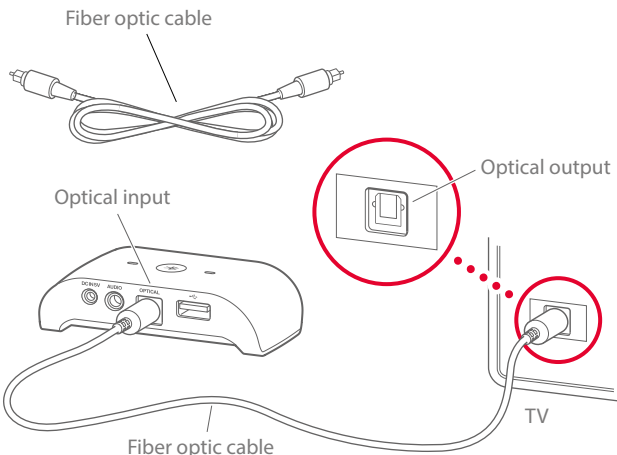
- 1 Connect one end of the cable to the optical input on the back of the TV streamer.
- 2 Connect the other end to the optical output on your TV.

Step 2: Connecting power

- 1 Connect the DC end of the power cord to the power input on the back of the TV streamer.
- 2 Connect the USB end to the Maxi power adapter.
- 3 Fit the adapter plug to the power adapter and connect it to a wall outlet. The power indicator lights up in green.

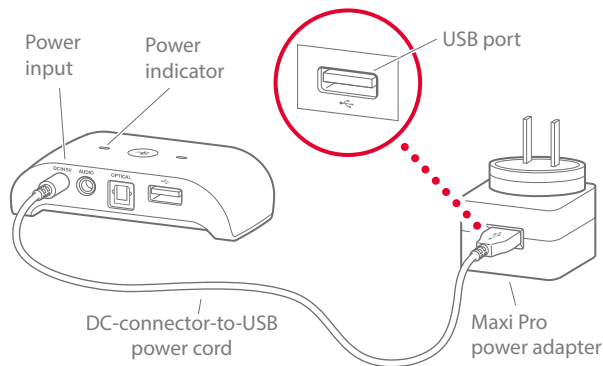You can also power it by connecting the USB end of the power cord to a TV USB port. This however might delay the TV sound in your headphones during reconnection.

Step 3: Pairing

Please note that if the TV streamer came with your Maxi Pro, you don't need to pair the units.

- 1 **On the TV streamer:** Press and hold the Bluetooth button until the Bluetooth indicator starts to blink in blue.
- 2 **On your Maxi Pro:** Press the microphone selector. Press and hold the Bluetooth selector until the Bluetooth indicator starts to blink in blue.
- 3 Both Bluetooth indicators will emit a steady blue light to show that the pairing was successful.

Using your TV streamer

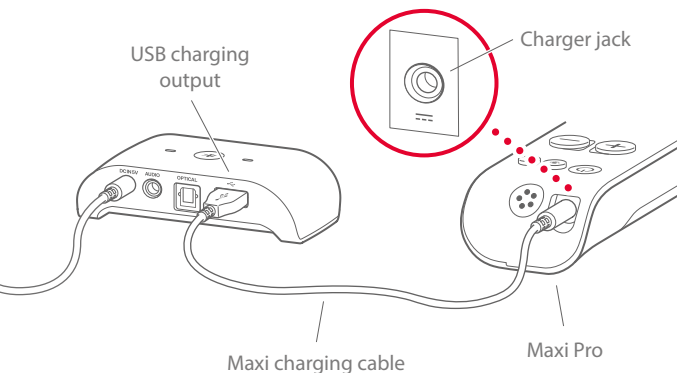
As long as your TV streamer is connected to power, it will remain on and automatically reconnect to your Maxi Pro when it returns within Bluetooth range. Both Bluetooth indicators will light up in blue when the units are connected.

If you power the TV streamer from your TV, it will start automatically when you turn on the TV and reconnect with your Maxi within 10 seconds.

Charging your Maxi Pro

Here is how you charge your Maxi from the TV streamer:

- 1 Connect the USB end of the Maxi charging cable to the USB charging output on the back of the TV streamer.
- 2 Connect the other end to the Maxi charger jack.
- 3 The Maxi battery indicator blinks in green during charging.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use.

If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Protect cables from any potential source of damage.
- Protect the device from shocks during storage and transport.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm, 12" to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- The power adapter must not be exposed to water drops or splashes and objects containing liquids, such as vases must not be placed on the unit. To disconnect the power fully, the adapter plug must be disconnected from the mains socket. The adapter plug must not be blocked and must be easy to access for the intended use. Rating information is shown on the rear.
- Do not block any of the ventilation openings. Install the device in accordance with the manufacturer's instructions.
- Do not defeat the safety purpose of the polarized or grounding - type plug. A polarized plug has two blades with one wider than the other. A grounding type plug has two blades and a third grounding prong. The wide blade or the third prong are provided for your safety. When the provided plug does not fit into your outlet, consult an electrician for replacement of the obsolete outlet.
- Protect the power cord from being walked on or pinched particularly at plugs and the point where they exit from the unit.
- Unplug this apparatus during lightning storms or when unused for long periods of time.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void the warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Only use power adapters and accessories specified in this booklet.

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not drop your device. Dropping onto a hard surface can damage it.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- Do not use the device in areas where electronic equipment is prohibited.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship.

This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Bluetooth® certified

This device includes a Bluetooth module which has passed the BQB certificate. The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by Bellman & Symfon is under licenses.

Product disposal and recycling information

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website bellman.com.

Symbol explanation

	Indicates the manufacturer's serial number so that a specific device can be identified.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates the device manufacturer.
	This symbol indicates the need for the user to consult the instructions for use.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	To identify a place where information may be found, especially in an emergency.
	Indicates the temperature limits to which the device can be safely exposed.
	Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If	Try this
Nothing happens when I press the TV streamer Bluetooth button.	Make sure that the TV streamer is connected to power, see Connecting power.
I don't hear the sound of the TV in my headphones.	Make sure Maxi Pro is within range and paired with your TV streamer, see Pairing.
The TV sound disappears when I connect the mini-stereo cable to the headphones output.	This is a feature on some TV models. Use the RCA or optical output on your TV instead, see Connecting audio.

EN

Technical specifications

Dimensions	Size: 88 × 24 × 48 mm, 3.5 × 0.9 × 1.9"
Weight	70 g, 2.5 oz.
Frequency	Bluetooth freq: 2400 – 2483.5 MHz
Coverage	Up to 25 m, 27.5 yd, free field. Connectivity: Qualcomm® aptX™ LL
Power	Power consumption: 31 mA
Connections	Audio input: 3.5 mm mini-stereo jack Audio input sensitivity @ max 2Vrms: +6 dBV Optical input: 44.1KHz, 48KHz Optical input sensitivity: 0 dBFS Charger input: 100 – 240V~, DC 5V, 1.0A Charging output: USB type A
Operation	Temp: 0° – 35 ° C, 32° – 95° F Humidity: 15% – 90%, non condensing
In the box	- BE2022 Maxi Pro TV Streamer - BE9123 Audio Cable kit Maxi Pro TV Streamer - BE9228 Fiber optic cable, white - BE9254 Charge cable USB, white
Accessories	- BE2021/BE2021-C/BE2021-P01 Maxi Pro Personal Amplifier - Power adapter, white - Power adapter plug, white

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JP

FCC compliance statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Users are not permitted to make changes or modify the device in any way. Changes or modifications not expressly approved by Bellman & Symfon Group AB will void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains license-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

This device may not cause interference.

This device must accept any interference, including interference that may cause undesired operation of the device.

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Revision: BE2022-01_003MAN3.0
Date of issue: 2025-01-13
TM and © 2025 Bellman & Symfon AB.
All rights reserved.

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Bellman & Symfon North America Inc.
Bellman & Symfon Asia Ltd.