
BE1470
Visit Pager Receiver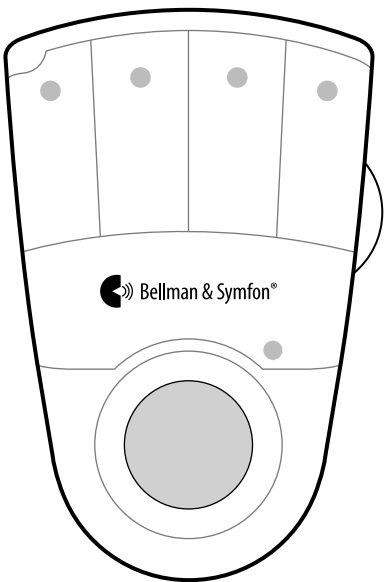

BE1470 Visit Pager Receiver

4 User manual

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EN

JP


Read this first

This booklet guides you on how to use and maintain your new Pager. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

Intended user group The intended user group consists of people of all ages.


Principle of operation The Visit system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview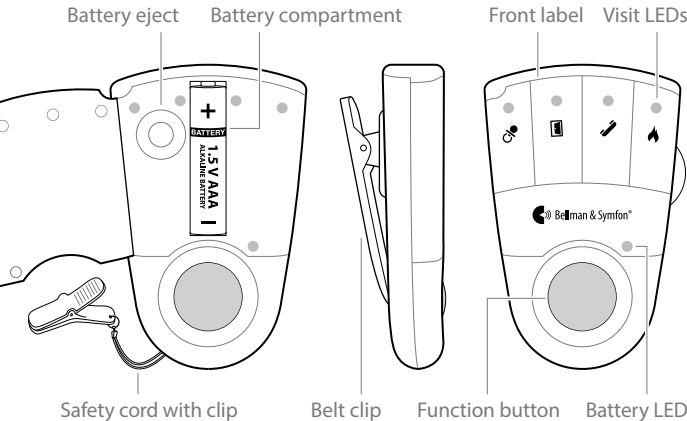

Getting started

- 1 Open the battery cover, fit the battery and close the cover again. Attach the Pager to your belt using the belt clip. For extra security, use the supplied safety cord.
- 2 To test the radio link you need a Visit transmitter. Press the test button/s on the transmitter.
- 3 The Pager starts to vibrate and lights up a Visit LED. If a Bed Shaker is connected during charging, it will vibrate. If nothing happens, see **Troubleshooting**.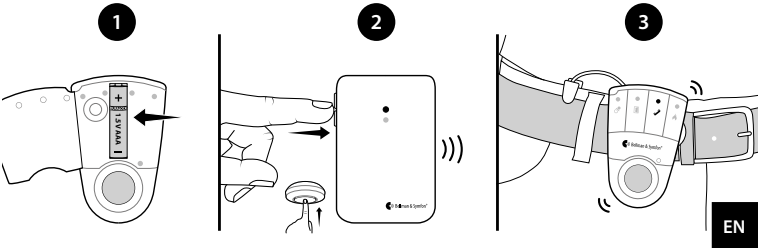

Default signal pattern

When a transmitter is activated, the pager lights up a Visit LED and starts to vibrate with a certain pace. This is called signal pattern. The transmitters determine the pattern, and the default is:

Transmitter	Pager LED	Vibration
■ Door Transmitter	Green	Slow ■ □ □ □
■ Push Button Transmitter	Green	Slow ■ □ □ □
■ Telephone Transmitter	Yellow	Medium ■ ■ □ □
■ Baby Monitor	Orange	Fast ■ ■ ■ ■ ■ ■
■ Smoke Alarm	Red	Long ■ ■ ■ □
■ CO Alarm	Orange + Red	Long ■ ■ ■ □

If the pager is docked to the charger accessory, the connected Bed Shaker will vibrate with the same pace as the Pager. Please note that the Pager will not vibrate when it's docked to the charger.

Personalizing your receiver

Changing the signal pattern

The signal pattern can only be changed on the transmitters. See 'Changing the signal pattern' for the relevant transmitter.

Changing the front label

If you want to use Visit for other purposes, the pager front label can be replaced with a customized one. Here is how it's done:

- Open the battery cover, replace the original label with the supplied extra label and close the cover again.

Replacing the battery

When the battery LED starts to blink in yellow, the battery is nearly depleted. Here is how you replace it:

- Open the battery cover and press the battery eject button to remove the old battery. Insert a 1.5 V AAA alkaline battery or a 1.2 V AAA NiMH rechargeable battery if you are using the BE1260 Pager Charger accessory.

Pager accessories


The pager can be complemented with the following accessories:

▪ BE1260 Pager Charger

Charges your pager during the night. Place it on the bedside table and connect up to two bed shakers. Please note that the pager will not vibrate when it's docked to the charger. The Visit LEDs will however light up as usual.

▪ BE1270 Bed Shaker

Wakes you with vibrations if anything happens while you are asleep. Connect it to the pager charger and slide it under your pillow or mattress.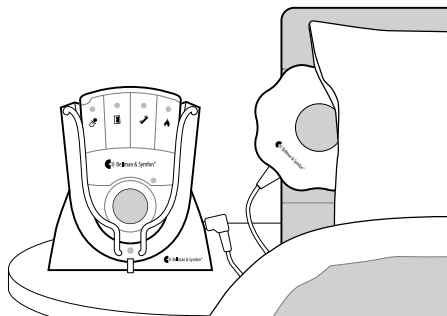


Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located on the **transmitters**.

To change the radio key:

- 1 Open the transmitter cover and move any radio key switch to the up (on position) to change the radio key. See '**Changing the radio key**' for the relevant transmitter.
- 2 Press and hold the function button on the pager until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the test button/s on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the pager blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note that all Visit units must be set to the same radio key in order to operate as a group.

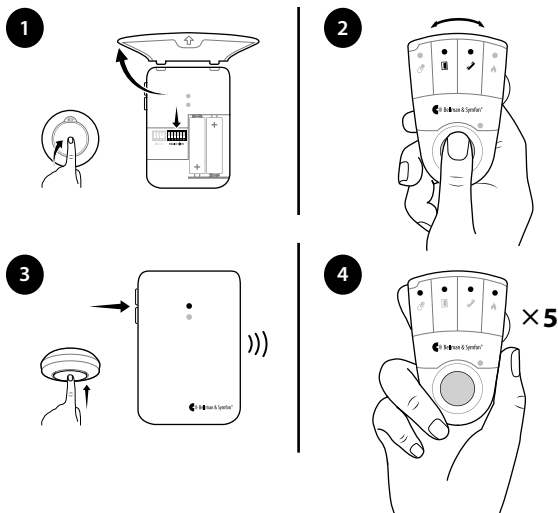Advanced programming

By using advanced programming, you can customize the signal pattern from a specific transmitter and event, displaying the LED colour and vibration pattern of your choice. The advanced programming overrides the radio key and pairs the units via the serial number. Please note that smoke alarms cannot be programmed for safety reasons.

Note: The transmitter must be activated as it is intended to be used in the system to generate the right signal. This means that you can't always use the transmitter test button, see **Default signal pattern** for the relevant transmitter.

Here is how you program the pager:

- 1 Press and hold the function button on the pager. The green and yellow Visit LEDs will start to blink alternately. While still holding down the button, activate the desired transmitter as intended. The yellow battery LED on the pager will light up to indicate that you are in advanced programming mode. Release the button.
- 2 Scroll through the different **Visit LED options** by pressing the function button on the pager. Select the desired Visit LED pattern by holding down the function button until the battery LED goes out and lights up again.
- 3 The pager will now show the new Visit LED colour and vibration pattern. Press the function button briefly to end the demonstration. After a short while, the pager will return to normal mode.

Deleting the advanced programming

Follow the procedure below to delete the advanced programming:

- 1 Hold down the function button on the pager until the green and yellow Visit LEDs blink alternately. Release the button.
- 2 Press the function button on the pager 3 times in quick succession.
- 3 All Visit LEDs will light up for ~2 seconds to show that it has been deleted.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications can be missed if batteries run out.
- This device should not be used as the only means of security in life-threatening situations.
- Use only power adapters and battery types that are specified in this booklet.
- **ONLY USE RECHARGEABLE NiMH BATTERIES** for the pager when you are using the pager charger. Non-rechargeable batteries will start to leak if the pager is placed in the charger and the battery acid will damage the electronics. The resulting damage is not covered by warranty.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only.
- Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- This device has no power switch. In order to disconnect the device, the battery must be removed.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power, or alter the original characteristics or performance on an approved low power radio frequency device.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for

two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If	Try this
The Pager seems to be turned off	<ul style="list-style-type: none">▪ The battery is depleted. Replace it with a 1.5V AAA alkaline battery. <p>Important! If you have a pager charger; only use a rechargeable 1.2 V AAA NiMh battery in the pager.</p>
The battery LED blinks in yellow	<ul style="list-style-type: none">▪ The battery level is low. Replace it with a 1.5V AAA alkaline battery. <p>Important! If you have a pager charger; only use a rechargeable 1.2 V AAA NiMh battery in the pager.</p>
The Pager does not respond when a transmitter is activated	<ul style="list-style-type: none">▪ Check the batteries in the transmitters.▪ Move the pager closer to the transmitter to make sure it's within radio range.▪ Check that the pager is set to the same radio key as the other units in the Visit system, see Changing the radio key on our website.
The Pager is activated for no apparent reason	<ul style="list-style-type: none">▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key on our website.

Technical specifications

Dimensions	86×57×29 mm, 3.4×2.2×1.1"
Weight	70 g, 2.5 oz. incl. battery
Mains power	Powered via the BE1260 Pager Charger accessory
Battery power	1×1.5 V AAA alkaline or 1×1.2 V AAA NiMH rechargeable battery
Operation time	Alkaline battery: 2 – 3 weeks NiMH battery: ~1 week, fully charged
Power consumption	Active: ≤220 mA Idle position: ≤1 mA
Radio frequency	315 MHz
Coverage	30 – 80 m depending on the characteristics of the building. Coverage is reduced if walls and large objects block the signal.
Operation	Temperature: 0° to 35° C, 32° to 95° F, Humidity: 15% to 90%, non-condensing
Configuration	This receiver can be used with all Visit transmitters.
Accessories	<ul style="list-style-type: none">▪ BE1260 Pager Charger▪ BE1270 Bed Shaker
In the box	<ul style="list-style-type: none">▪ BE1470 Visit Pager Receiver▪ Safety cord with clip▪ Extra front label

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JP

Model, type and classification

The information is available at the back of the product.

Product disposal and recycling information

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

The product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

Bellman & Symfon Asia Ltd.
1401 & 1408, Laifung International Centre
33 Jie Fang Road South
Guangzhou, China
Phone: +86 20 8318 2950

DESIGN FOR EARS™

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim, Sweden
Phone: +46 31 68 28 20
E-mail: info@bellman.com
bellman.com

Revision: BE1470-315_001MAN2.0
Date of issue: 2025-01-13
TM and © 2025 Bellman & Symfon AB.
All rights reserved.