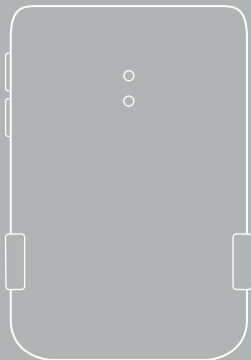
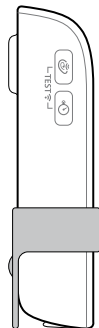
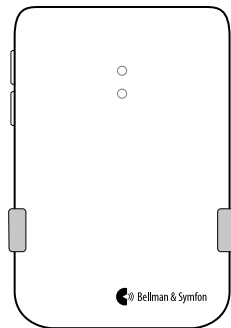
BE1491
Visit Baby Monitor

Visit Baby Monitor

- | | |
|----|-------------------------|
| 4 | User manual |
| 18 | Manual del usuario |
| 32 | Manuel d'utilisation |
| 46 | 用户手册 |
| 60 | 用戶手冊 |
| 75 | ユーザーマニュアル |
| 88 | Appendix (English only) |

EN

ES

FR

CN_{SI}

CN_{TM}

JP

App

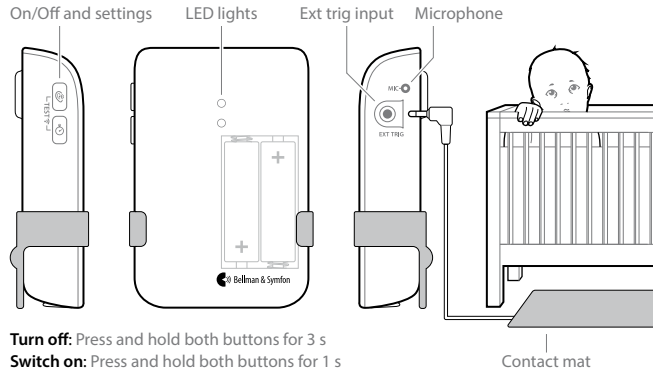
Read this first

This booklet guides you on how to use and maintain your new Baby Monitor. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

Intended user group The intended user group are people of all ages.


Principle of operation The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview


Installation

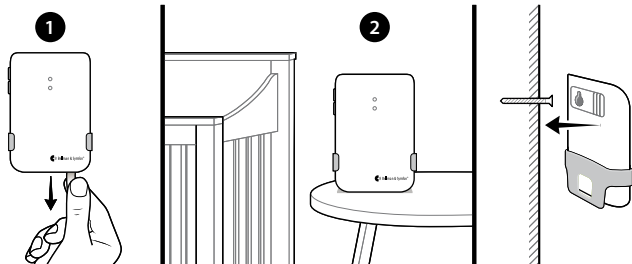- 1 Remove the battery pull tab to start the baby monitor.
- 2 Place the baby monitor on the nightstand or hang it on the wall with the screw and plug. The recommended distance is 0.5 - 2 m, out of your baby's reach.

Testing

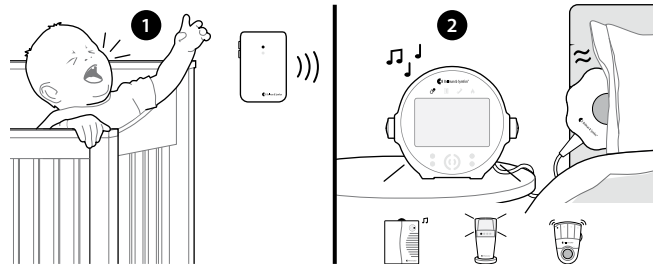- 1 Stand by your baby's bed and make some noise.
- 2 The orange Visit LED on the receiver lights up and it starts to sound, flash or vibrate, depending on the receiver. If it doesn't react, read **Settings**.

Settings

The buttons for sensitivity and delay are located on the baby monitor's left side. When you press the button, the LED shows the current setting. Then press repeatedly to adjust the setting.

- If the baby monitor isn't activated when your baby cries – increase the sensitivity.
- If the baby monitor is activated too easily – reduce the sensitivity.
- If the baby monitor is activated too fast or too slow – adjust the delay.


Sensitivity

Red LED	65 dB
Orange LED	75 dB
Green LED	85 dB


Delay

Red LED	30 s
Orange LED	10 s
Green LED	1 s

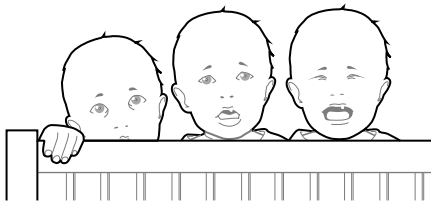Installing a contact mat

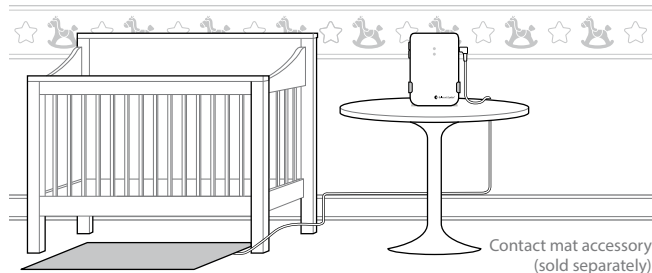By using a contact mat, you can also be notified if your baby gets out of bed. Just place the mat in front of the bed, connect it to the baby monitor and it will tell you when your baby steps on it.

Contact mat accessory
(sold separately)

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years. Never place the device in the child's crib or playpen.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow the instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty

last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

 SN	Indicates the manufacturer's serial number so that a specific device can be identified.		Identifies a place where information may be found, especially in an emergency.
	Indicates the device manufacturer.		Indicates the need for the user to consult the instructions for use.
	Indicates that caution is necessary when operating the device close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.		Indicates the temperature limits to which the device can be safely exposed.
			Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If	Try this
Two orange lights appear every 10 seconds	▪ The batteries are depleted. Replace them with 1.5 V AA (LR6) lithium or alkaline batteries.
A green light appears on the baby monitor but the receiver is not activated	▪ Check the baby monitor batteries and the receiver batteries and connections. ▪ Move the receiver closer to the baby monitor to make sure it's within range. ▪ Check that the units are set to the same radio key, see Changing the radio key in the Appendix (English only).
The baby monitor is not activated even though the baby is crying	▪ Move the baby monitor closer to the baby or increase the microphone sensitivity, see Settings. ▪ Warning: Always make sure the baby monitor is out of your baby's reach. Never place the baby monitor in the child's crib or playpen.

The baby monitor is activated too easily	▪ Reduce the sensitivity or move the baby monitor further away from the crib, see Settings.
The baby monitor is activated too early	▪ Increase the delay, see Settings.
The baby monitor is activated too late	▪ Reduce the delay, see Settings.
The receiver is activated for no apparent reason	▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key in the Appendix (English only).

Technical specifications

Dimensions	100 × 65 × 27 mm, 4.0"× 2.6"× 1.1"
Weight	120 g, 4.2 oz. including battery
Power and battery	Battery power: 2 x 1.5 V AA (LR6) lithium or alkaline batteries. Power consumption: Active < 70 mA, Idle position < 400 µA Operating time: Alkaline ~ 6 months, Lithium ~ 1 year
Activation	Via the test buttons, microphone or a contact mat via the ext. trig.
Radio frequency	433.92 MHz/314.90 MHz
Coverage	70 - 120m, 230-394 ft (433Mhz) / 30-80m (315Mhz), depending on the building's characteristics.
Number of radio keys	64 as standard. Up to 256 with optional software.
Operation	Temp: 0° – 35° C, 32° – 95° F, Humidity: 15% – 90%, non-condensing
Configuration	This transmitter can be used with all Visit receivers.
Accessories	▪ BE9026 Contact mat
In the box	▪ BE1491 Visit Baby Monitor ▪ 2 x 1.5 V AA (LR6) Alkaline batteries ▪ Table stand

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
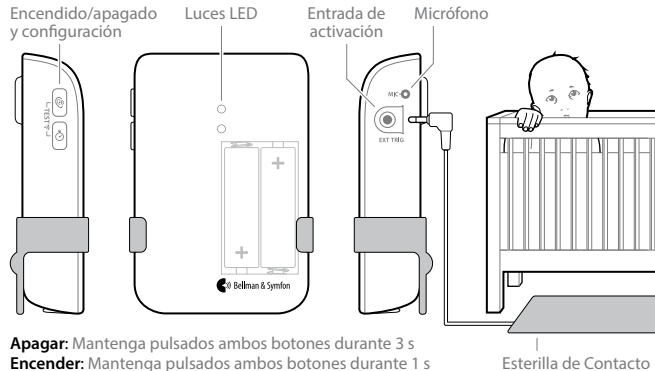
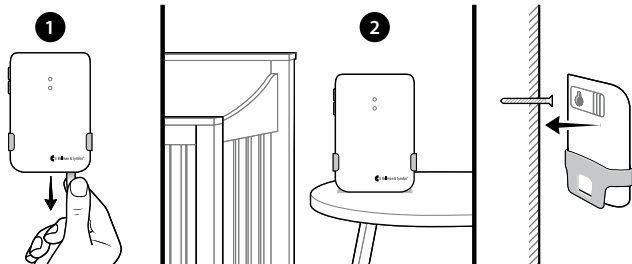
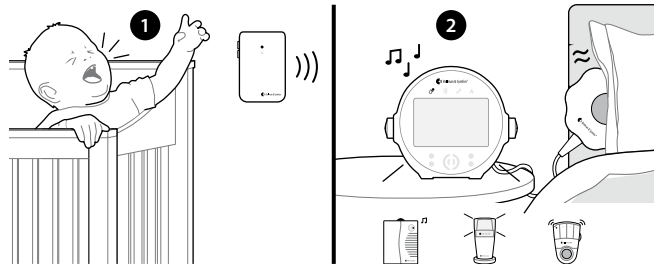
Sensibilidad

LED rojo	65 dB
LED naranja	75 dB
LED verde	85 dB

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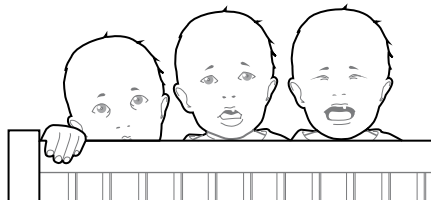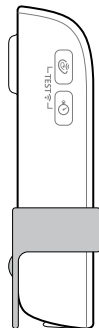
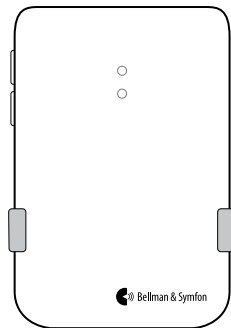
Appendix (EN)

- 90 Testing the signal
- 91 Default signal pattern
- 92 Changing the signal pattern
- 94 Changing the radio key

Testing the signal

Testing the baby monitor is easy. First, make sure that the baby monitor is installed properly. For instructions, read the section **Installation**.

Using the test button

Press both test buttons simultaneously, see **Overview** for button location.

- 1 The baby monitor's LED lights up to show that a radio signal is transmitted.
- 2 The receiver's pacifier symbol lights up to show that the signal was received.

Using the microphone

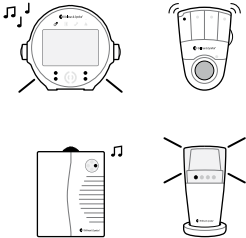
Activate the baby monitor by whistling or speaking into the microphone.

- 1 The baby monitor's LED lights up to show that a radio signal is transmitted. If you have adjusted the delay, it will be postponed as shown in **Settings**.
- 2 The receiver's pacifier symbol lights up to show that the signal was received. If you have customized the signal pattern, it will blink as shown in the signal pattern tables on the following spreads.

Default signal pattern

When the baby monitor is activated by the baby's voice or the contact mat, the following happens:

- 1 The LED on the baby monitor lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in orange and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The baby monitor and the contact mat accessory determine the signal pattern. The default signal pattern is according to the table below:

Baby monitor


Source	LED
Baby voice	Green, top
Contact mat	Green, bottom

Visit receiver

LED	Sound	Vibration	Flash
Orange light	Baby melody	Yes	Yes
Green light	Door chime, low	Yes	Yes

Changing the signal pattern

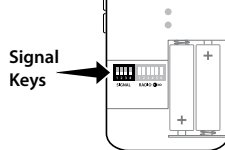If you have more than one baby, you can set a unique Visit LED pattern for each baby monitor. Just open the front cover and move any signal switch to the up (on position) and make sure the other switches are down (off).

Baby monitor		Visit receiver		315 MHz	
Switch	Source	LED	Sound	Vibration	Flash
	Baby monitor 1 Contact mat 1	Orange light Green light	Baby melody 1x Door chime low	Fast Slow	Yes Yes
	Baby monitor 2 Contact mat 2	2 × orange blinks 2 × green blinks	Baby melody 2x Door chime low	Fast Slow	Yes Yes
	Baby monitor 3 Contact mat 3	3 × orange blinks 3 × orange blinks	Baby melody Baby melody	Fast Fast	Yes Yes
	Baby monitor 4 Contact mat 4	Orange blinks Orange blinks	Baby melody Baby melody	Fast Fast	Yes Yes

Baby monitor		Visit receiver		433 MHz	
Switch	Source	LED	Sound	Vibration	Flash
	Baby monitor 1 Contact mat 1	Orange light Green light	Baby melody 1 x door chime, low	Long Long	Yes Yes
	Baby monitor 2 Contact mat 2	Orange light Green light	2x ring signal, low 1x ring signal, high	Medium Medium	Yes Yes
	Baby monitor 3 Contact mat 3	Orange light Orange light	2x door chime, low 2x door chime, low	Long Long	Yes Yes
	Baby monitor 4 Contact mat 4	Orange blinks Orange blinks	Baby melody Baby melody	Long Long	Yes Yes

App

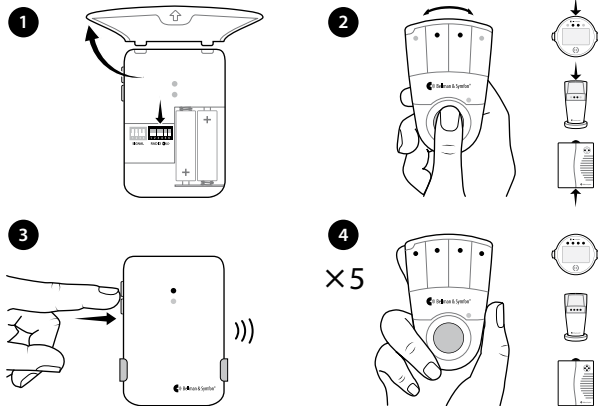
Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the monitor cover.

Here is how you change the radio key:

- 1 Carefully remove the table stand and open the front cover on the baby monitor. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the baby monitor within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink alternately to show that the radio key has been changed. It then returns to normal mode. If you have a BE1033 Portable Receiver, the power LED blinks in amber to confirm.

Please note: All Visit units must be set to the same radio key.

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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FCC ID: WMSBETXBC **IC:** 6693A-BETX3

Product disposal and recycling information

According to local laws and regulations your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

This product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

Bellman & Symfon North America Inc.
PO Box 1388, Morrisville,
North Carolina 27560, USA
Phone: (910) 239-5796
Customer support: (833) BELLMAN
or (833) 235-5626

Bellman & Symfon Asia Ltd.
1401 & 1408
Laifung International Centre
33 Jie Fang Road South
Guangzhou, China
Phone: +86 20 8318 2950

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1491_127MAN2.0

Date of issue: 2025-01-29

TM and © 2024 Bellman & Symfon AB.

All rights reserved.