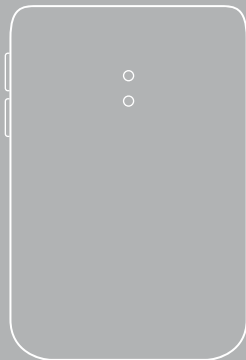
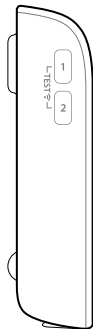
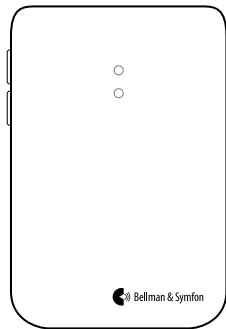
BE1431-868
Visit Telephone Transmitter

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
Read this first

This booklet guides you on how to use and maintain your new Visit Telephone Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.


- Intended purpose** The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.
- Intended user** A person with mild to severe hearing loss or deafness.
- Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.
- Principle of operation** The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

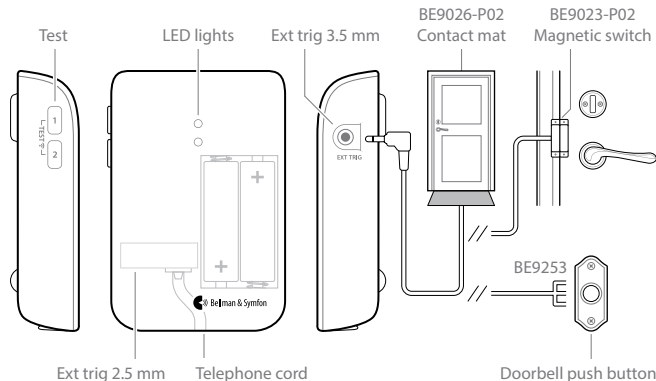Overview


Installation

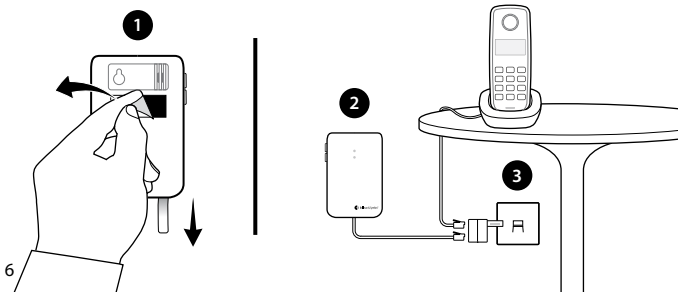- 1 Remove the battery pull tab to start the unit. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the transmitter on the wall. You can also use the supplied screw and plug.
- 3 Connect the telephone splitter between the telephone plugs and the socket.

Testing

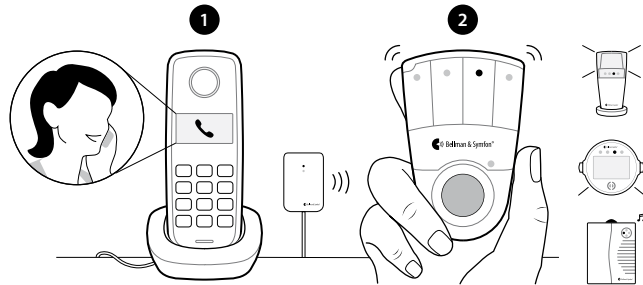- 1 Use your mobile phone or ask a friend to call your landline telephone.
- 2 The yellow Visit LED on the receiver lights up and it starts to sound, flash or vibrate, depending on the receiver. If it doesn't react, read **Troubleshooting**.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.

- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.

- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

MD

Indicates the item is a medical device.

UDI

Indicates a carrier that contains unique device identifier information.

Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.

SN

Indicates the manufacturer's serial number so that a specific medical device can be identified.

REF

Indicates the manufacturer's catalogue number so that the medical device can be identified.

Indicates the medical device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the medical device can be safely exposed.

Indicates the range of humidity to which the medical device can be safely exposed.

Troubleshooting

If

Try this

The LEDs blink in orange every minute

- The batteries are depleted. Replace them with 1.5 V AA (LR6) lithium or alkaline batteries.

The transmitter LED lights up in green but the receiver doesn't respond

- Check the batteries and connections on all Visit units.
- Move the receiver closer to the transmitter to make sure it's within radio range.
- Check that the units are set to the same radio key, see **Changing the radio key** on our website.

The transmitter LED doesn't light up when the phone rings or when an accessory is triggered

- Press the test buttons. If the LED lights up in green, check all connections. If the LED doesn't light up, replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.

If	Try this
The transmitter LED doesn't light up when I press the test buttons	Replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries. If the LED still doesn't light up, contact your retailer for service information.
The receiver is activated for no apparent reason	There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key on our website.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website **bellman.com**

Website:

Technical specifications

Dimensions	100 × 65 × 27 mm, 4.0 × 2.6 × 1.1"
Weight	134 g, 4.7 oz. including battery and telephone cord
Power and battery	Batteries: 2 × 1.5 V AA (LR6) lithium or alkaline Power consumption: Active < 70 mA, Idle position < 15 µA Operating time: Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via test buttons, RJ11 input to analogue telephone network (26-120V RMS, 15-100Hz), 2.5 mm ext trig, 3.5 mm ext trig
Max. output power	4.7 dBm
Radio frequency	868.30MHz
Coverage	50 - 250 m depending on the characteristics of the building.
No. of radio keys	64 as standard. Up to 256 with optional software.
Operation	Temperature: 0° to 35° C (32° to 95° F), Humidity: 15% – 90%, non-condensing
Configuration	This transmitter can be used with all Visit receivers
In the box	- BE1431-868 Visit Telephone Transmitter 1.5V AA alkaline batteries - Velcro, screws and plugs
Accessories	- BE9026-P02 Contact mat - BE9253 External trig cable - BE9023-P02 Magnetic switch

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM and © 2025 Bellman & Symfon AB.