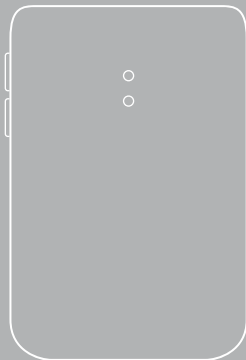
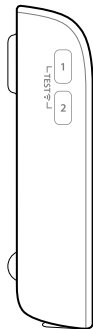
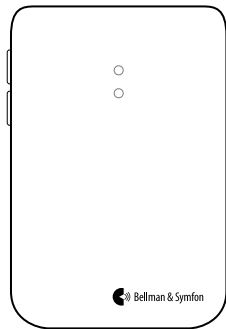
BE1411-868
Visit Door Transmitter

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EN
DE
DK
ES
FI
FR
IT
NL
NO
PL
SE
SK

Read this first

This booklet guides you on how to use and maintain your new Visit Door Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

Intended purpose

The intended purpose of the Visit system is to alert deaf and hard of hearing people of important signals in their home.

Intended user

The intended user is a person with mild to severe hearing loss or deafness.

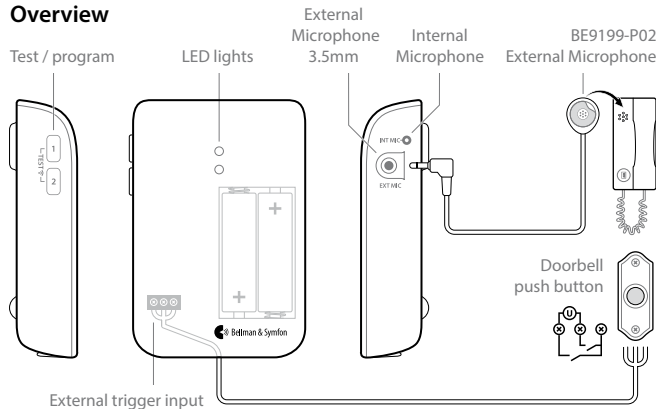
Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

Principle of operation


The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview


Basic installation – single sound source

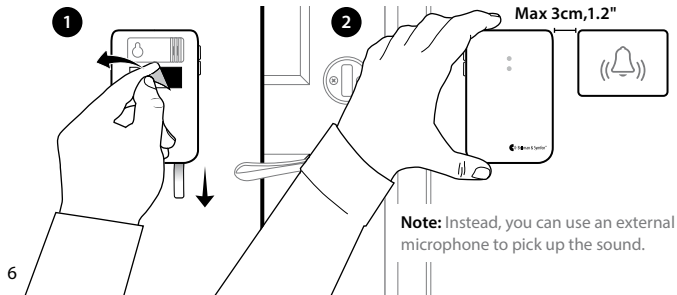- 1 Remove the battery pull tab to start the unit. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. You can also use the supplied screw and plug.

Installation – intercom and electronic doorbell

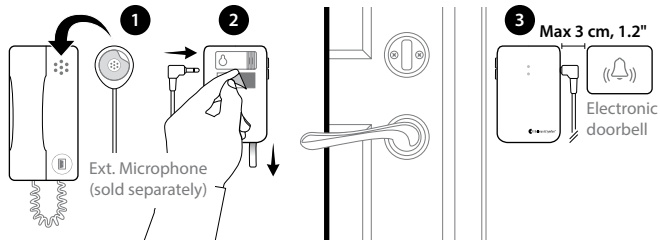
- 1 Remove the protective film from the external microphone (sold separately) and attach it to the intercom's speaker. Connect it to the transmitter.
- 2 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 3 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. Note that the intercom and doorbell must be at least 25 cm, 10" apart to avoid sound interference.

Installation – intercom and mechanical doorbell

If you have an intercom and a mechanical doorbell, you may need two door transmitters, i.e. one for each sound source.

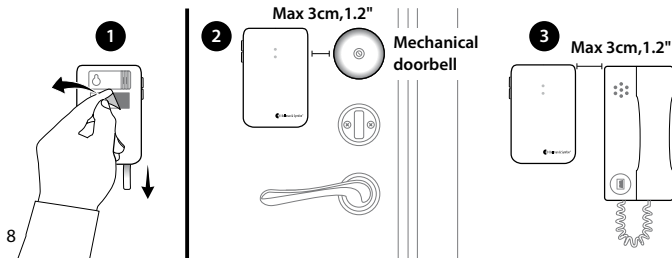
- 1 Pull the battery tab to start each door transmitter. Clean the wall surface with the wet wipe and remove the protective film from the Velcro.
- 2 Mount one of the door transmitters to the left of the doorbell and the other to the left of the intercom's speaker, as close to the sound sources as possible.

Optimizing the doorbell detection


The next step is to teach the transmitter to recognize your doorbell.

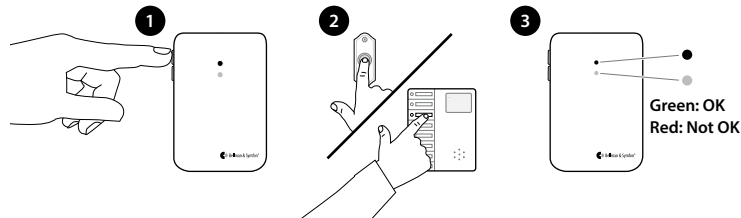- 1 Press and hold button 1 until the top LED start to blink. Release it to start the recording.
- 2 **For door buzzers:** Press and hold the doorbell for at least 8 seconds.
For door chimes: Press the doorbell at least 8 times.
- 3 Press button 1 on the transmitter to stop the recording. If the LED lights up in red, you need to repeat steps 1 and 2. Following the same steps, you can use button 2 to record the sound of your intercom.

Testing the signal

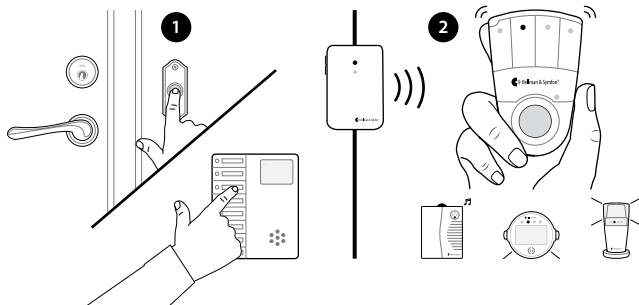- 1 Ring the doorbell or intercom.
- 2 The LED on the Visit receiver lights up in green and the receiver starts to sound, flash or vibrate, depending on the model. If it doesn't react, see **Troubleshooting**.

Sound Level Mode (The transmitter only reacts to dB levels.)

1 Prepare the device

- Remove the front cover and one of the batteries.
- Wait for 10 seconds.
- Hold down test button 1 and 2 simultaneously.
- While still holding down the buttons, refit the battery.

2 Activate Sound Level mode

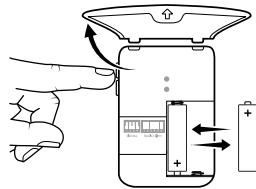
- Keep holding down the test buttons until the green LEDs light up.
- Release the buttons.
- The unit is now in Sound Level Mode.

3 Choose a sensitivity

- Use the test buttons to toggle between the sensitivity thresholds: Red: 65dB, Amber: 75dB and Green: 85dB

To deactivate Sound Level mode

- Repeat the steps in "Prepare the device".
- Release the buttons once the battery is refitted. The LEDs light up in green to confirm.

Please note: Sound Level Mode may generate unwanted alerts caused by ambient noise.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.

- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.

- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

MD

Indicates the item is a medical device.

UDI

Indicates a carrier that contains unique device identifier information.

Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.

SN

Indicates the manufacturer's serial number so that a specific medical device can be identified.

REF

Indicates the manufacturer's catalogue number so that the medical device can be identified.

Indicates the medical device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the medical device can be safely exposed.

Indicates the range of humidity to which the medical device can be safely exposed.

Troubleshooting

If

Try this

The LEDs blink in orange every minute

- Change the batteries. Only use 1.5 V AA (LR6) Lithium or Alkaline batteries.

The LEDs blink in orange every second

- There are other competing sound sources around the door transmitter. Switch them off or turn them down.
- Disconnect the External microphone to make sure it is not faulty.

The LED on the transmitter lights up when I press the doorbell or intercom - but the receiver is not activated

- Check the transmitter batteries and the receiver batteries and connections.
- Move the receiver closer to the transmitter to make sure it's within radio range.
- Check that the door transmitter and the receiver are set to the same radio key. For more information, see **Changing the the radio key** on our website.

The receiver is activated for no reason

- There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see **Changing the radio key** on our website.

EN

If	Try this
The LED on the transmitter doesn't light up when I ring the doorbell	- Make sure you have mounted the transmitter to the left of the doorbell sound source, no more than 3 cm apart. - If you are using an external microphone, make sure its connected. If you are using an intercom, make sure that the distance between the intercom and transmitter is at least 25 cm or that the internal microphone is turned off. - Try again to teach the transmitter to recognize the sound of your doorbell, see Optimizing the doorbell detection. - If the doorbell signal varies a lot, change to magnetic detection, see Electromagnetic detection on our website.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive 2014/53/EU.

The full text of the declaration of conformity can be obtained from the Bellman & Symfon website **bellman.com**.

Website:

Technical specifications

Dimensions	100×65×27 mm, 4.0×2.6×1.1"
Weight	100–118 g, 3.5–4.2 oz. depending on batteries
Power and battery	Battery: 2×1.5 V AA (LR6) Lithium or Alkaline Power consumption: Active < 70 mA, Idle position < 15 µA Operating time: Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via the test buttons, the internal or external microphone, via electromagnetic detection or the external trigger input
Radio frequency	868.30MHz
Max output power	4.7dBm
Coverage	50 - 250 m depending on the characteristics of the building
No. of radio keys	64 as standard. Up to 256 with optional software
Operation	Temperature: 0° to 35° C, Humidity: 15% - 90%, non-condensing
Configuration	This transmitter can be used with all Visit receivers
In the box	- BE1411-868 Visit Door Transmitter 1.5V AA Alkaline batteries - Velcro, screws and plugs
Accessories	BE9199-P02 External Microphone 2.5 m

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM and © 2025 Bellman & Symfon AB.