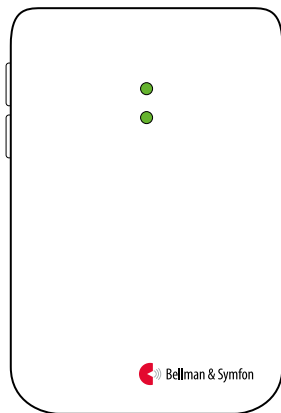
BE1433

Visit Mobile Phone Transceiver

BE1433

Visit Mobile Phone Transceiver

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EN
DE
DK
ES
FI
FR
IT
NL
NO
PL
SE
SK

Read this first

This booklet guides you on how to use and maintain your new Visit Mobile Phone Transceiver. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

Intended purpose

The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.

Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

Intended user

The intended user is a person with mild to severe hearing loss or deafness.

Principle of operation

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.


Let's get started!

What you'll need

- An Apple- or Android device with iOS 15 / Android 8 or later.

What's in the box

- | | |
|----------------------------|-----------------------|
| 1 Mobile Phone Transceiver | 4 Screw and plug |
| 2 Power cable | 5 Telephone splitter* |
| 3 Telephone cord | 6 Power adapter |

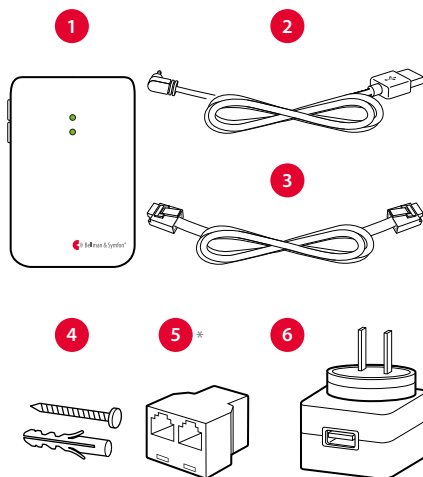*The inclusion and appearance of a phone adapter / splitter vary with region and package.

Installing the transceiver

1 Mount it

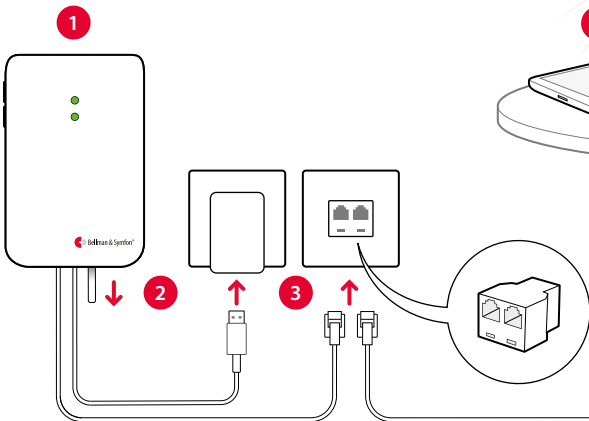
Remove the protective film from the Velcro on the back of the transmitter and mount it on the wall. Alternatively use the supplied screw and plug.

2 Turn it on

Pull the battery tab to start the transmitter. The indicators light up in green to show that it is on.

3 Connect it

Connect the USB cable to the power adapter. Then plug the adapter into a wall outlet. If you have a landline telephone, connect it to the telephone jack using the supplied splitter.


4 Test the mobile phone

Have someone call your mobile. The yellow LED on the Visit receiver blinks and it starts to flash, sound or vibrate (depending on the receiver).

5 Test the landline phone


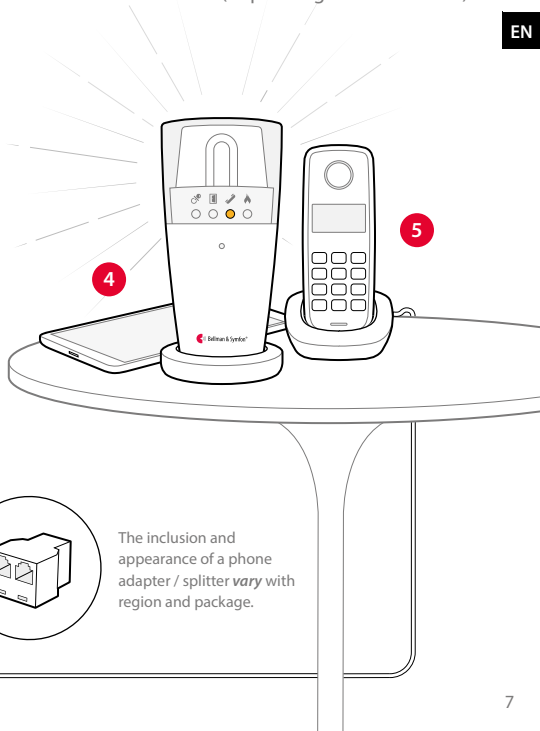
Call your landline phone. The yellow LED on the Visit receiver lights up and it starts to flash, sound or vibrate (depending on the receiver).

Installing the app

1 Turn on Bluetooth®

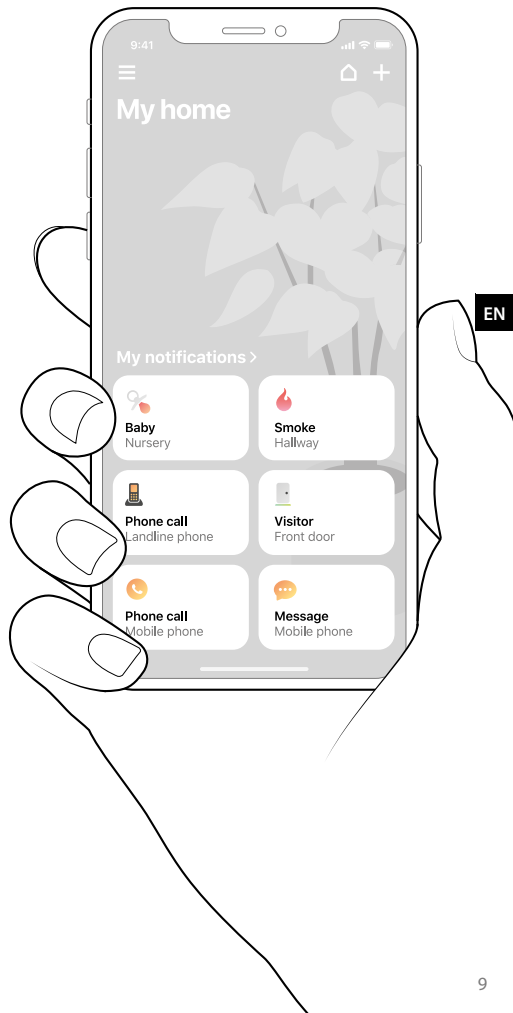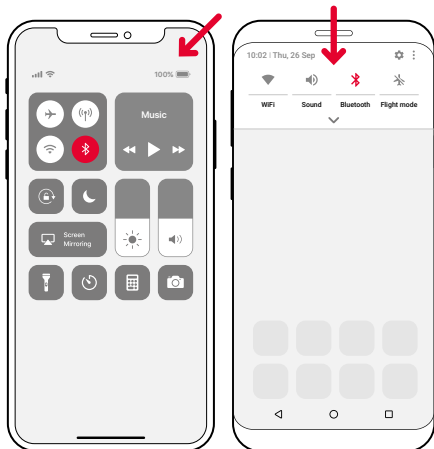Swipe down from the top of the screen and tap the Bluetooth icon.

2 Install the Visit app

Download the **Bellman Visit** app from the App Store® or Google Play™ and follow the setup instructions.

Using the app

Once you have installed the Visit app, you will start to receive Visit notifications on your mobile device. In addition, your Visit receiver will alert for mobile calls and messages. Controlling your system from the app is easy. Here's what you can do:

Manage notifications

Enable, disable or delete a notification with an easy tap.

Control your devices

Get an overview of your connected devices and tap + to add a new.

View history

See your latest notifications, sorted by date and time.

Learn more

Tap **User guide** in the menu to learn more about the features.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- This product uses Alkaline batteries as power backup. Do not expose the batteries to fire or to direct sunlight. Only use 1.5 V AA alkaline batteries.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void guarantee.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only power adapters and accessories that are specified in this booklet.

- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the guarantee.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and guarantee.

Guarantee conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Further the guarantee does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This guarantee gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied guarantee last so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above guarantee may not be altered except in writing signed by both parties hereto.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Bellman & Symfon is under license.

Technical specifications

Dimensions	100×65×27 mm, 4.0×2.6×1.1"
Weight	198 g, 6.98 oz. including batteries, telephone cable and power cable
Mains power	5 V DC / 1.0 A
Backup batteries	2×1.5V AA Alkaline
Connectivity	RJ11 input to analogue telephone network, 3.5 mm ext. trig input, Power supply jack, Bluetooth Low Energy, BLE
Frequency	ISM frequency: 868.30 MHz Transmission power: 4mW (max.) Bluetooth freq: 2400-2483.5 MHz Transmission power: 4mW (max.)
Coverage	ISM: Up to 250 m, depending on the characteristics of the building. Bluetooth: Up to 40 m
Compatibility	iOS15 and later / Android 8 and later iPhone 6S and later / Modern Android devices
Configuration	This transceiver can be used with all Visit receivers.
Operation	Temperature: 0° – 35° C, 32° – 95° F Humidity: 15% – 90%, non condensing
In the box	<ul style="list-style-type: none">▪ BE1433 Visit Mobile Phone Transceiver (including batteries)▪ Screw and plug▪ Power adapter and power cable▪ Telephone cord▪ Telephone splitter*
Accessories	<ul style="list-style-type: none">▪ BE9026-P02 Contact mat▪ BE9023-P02 Magnetic switch▪ BE9253 External trigger cable▪ BE9002 SS-RJ11 phone splitter (SE) (-T01)▪ BE9005 RJ11 phone splitter (-T02)▪ BE9007 BS-RJ11 phone splitter (UK) (-T03)▪ BE9008 F-010-RJ11 phone adapter (FR) (-T04)▪ BE9010 Tripolar-RJ11 phone splitter (FI) (-T05)

*The inclusion and appearance of a phone adapter / splitter vary with region and package.

Symbol explanation

	Indicates the item is a medical device.
	Indicates a carrier that contains unique device identifier information.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Indicates the medical device manufacturer.
	This symbol indicates the need for the user to consult the instructions for use.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	To identify a place where information may be found, especially in an emergency.
	Indicates the temperature limits to which the medical device can be safely exposed.
	Indicates the range of humidity to which the medical device can be safely exposed.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website bellman.com.

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM & © 2026 Bellman & Symfon AB.