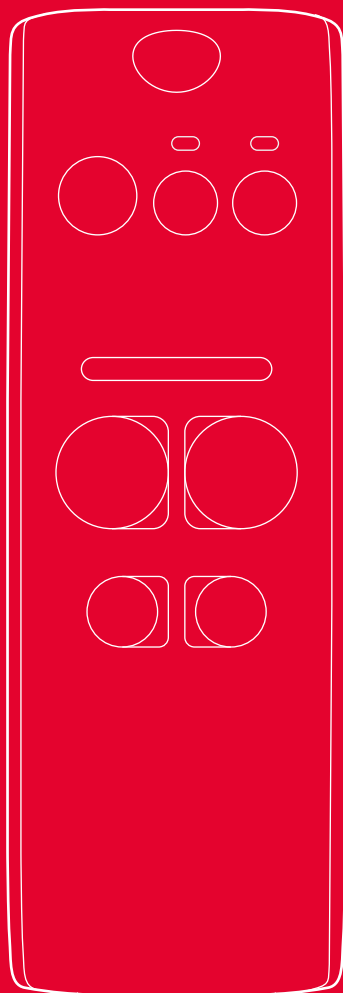
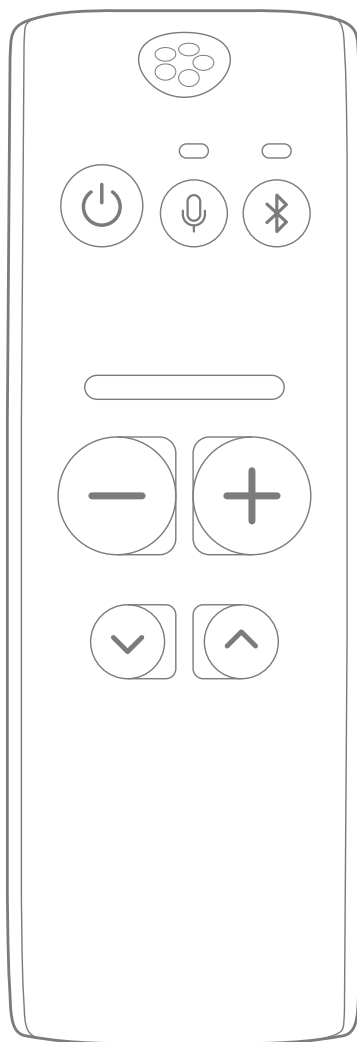
BE2021
Maxi Pro Personal Amplifier

BE2021 Maxi Pro Personal Amplifier

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JP

Read this first

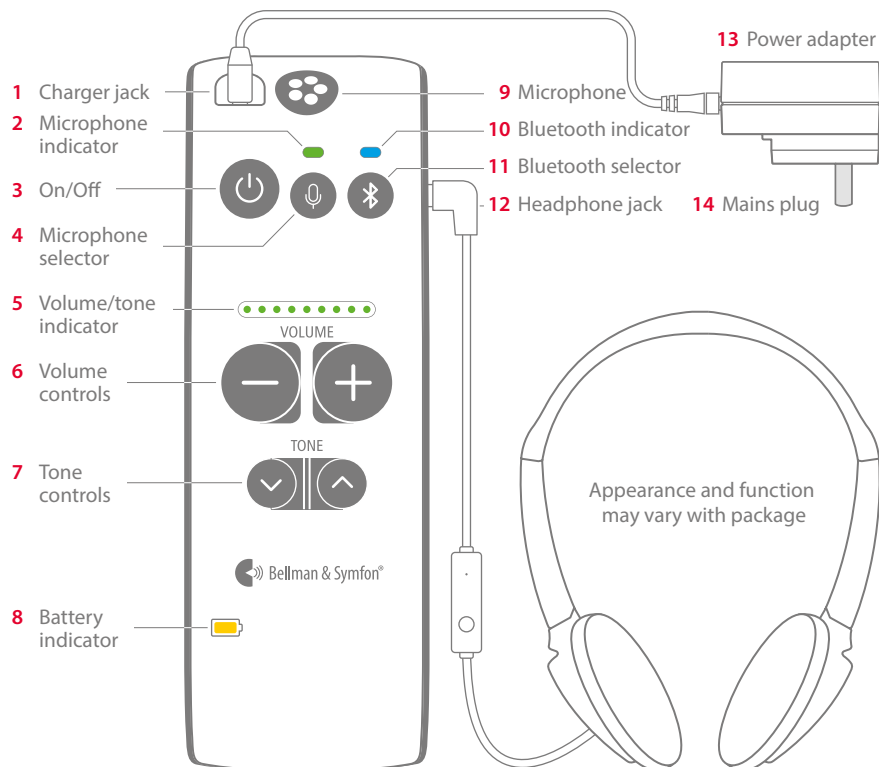
This booklet guides you on how to use and maintain your new Maxi Pro Personal Amplifier. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

Intended user group People of all ages.

Intended user A user that wants to control tuning and listening volume.

Principle of operation The audio product family consists of several amplifiers and sound transmitters which have been specifically developed to provide sound enhancement even in demanding situations. Depending on the assigned function of the specific amplifier or sound transmitter, different microphones can be used to pick up direct sound or enhance the ambient sound.

Overview

Getting started

Step 1: Charge Maxi

- 1 Connect the charging cable to the power adapter (13) and to the charger jack (1).
- 2 Fit the mains plug (14) to the power adapter (13) and connect it to the power outlet.
- 3 The battery indicator (8) blinks in green during charging. After about three hours it emits a steady green light to show that Maxi is fully charged.

Step 2: Connect Maxi

Connect the supplied headphones, earphones, headset, earset or neck loop to the Maxi headphone jack (12) marked in green.

Step 3: Start Maxi

Press the On/Off button (3) for about one second to start Maxi. The microphone indicator (2) lights up in green and the current volume is shown by the volume/tone indicator (5) for three seconds.

To switch Maxi off, press and hold the On/Off button (3) for about three seconds until all indicators go out.

Using Maxi to clarify speech

Maxi is equipped with a built in omni-directional microphone (9) that enhances speech. Place Maxi on a table close to the person talking, hold it in your hand or attach it to your clothes using the clip.

Adjusting the volume

Press **+** on the volume controls (6) to increase the volume and **-** to decrease it. The microphone volume can be adjusted in 9 steps and the TV and mobile phone volume in 15 steps. The level is shown in the volume/tone indicator (5).

Every time you turn on Maxi, the volume is set to low to avoid unpleasant sound levels.

Adjusting the tone

Press the **^** on the tone controls (7) to increase the treble and the **v** to decrease it.

The treble can be adjusted in 5 steps and is visible in the volume/tone indicator (5).

Increasing the treble makes the S, F and T sounds more pronounced and the words easier to make out.

Using Maxi with your mobile phone

Pairing

- 1 Make sure your Maxi is in microphone mode by pressing the microphone selector (4).
- 2 Press and hold the Bluetooth selector (11) until the Bluetooth indicator (10) starts to blink in blue.
- 3 Make sure Bluetooth is activated on your mobile phone. Open the Bluetooth menu in Settings and select "Maxi Pro" in the list.
- 4 The Bluetooth indicator will emit a steady blue light to show that the pairing was successful.

If your phone has been disconnected from Maxi, you might need to reconnect it by repeating step 3 to 4.

Answering a phone call

- 1 When the mobile phone rings, the Maxi Bluetooth indicator (10) blinks in blue. All other sounds are muted and you'll hear the ring signal in your headphones.
- 2 To **answer** the call, press the Maxi Bluetooth selector (11) or use the remote on your headset. You will now hear the enhanced voice of the caller in your headphones. Your voice will be picked up by the headset or the Maxi microphone.
- 3 To **hang up**, press the Bluetooth selector once more.

To **decline** an incoming call, press and hold the Maxi Bluetooth selector for 3 seconds.

Making a phone call

When you talk on the mobile phone, you hear the enhanced voice of the recipient in your headphones. Your voice will be picked up by the Maxi microphone.

Streaming music

Play a song on your mobile phone and press the Maxi Bluetooth selector (11) to hear the music. A second press could be needed if another Bluetooth source also is connected.

Using a headset with microphone

When you talk over the phone, the sound will be picked up from the headset microphone instead of the Maxi microphone.

Using Maxi for TV listening

In order to stream the sound from your TV, you need a Maxi Pro TV streamer, see separate user manual for installation instructions.

Listening to the TV

Turn on the TV, select a show and press the Maxi Bluetooth selector (11) to hear the TV. A second press could be needed if another Bluetooth source also is connected.

If you get a call during the show, the sound from the TV will be muted, so you can hear the caller's voice clearly. When you hang up, the sound from the TV will automatically come back.

Automatic reconnection

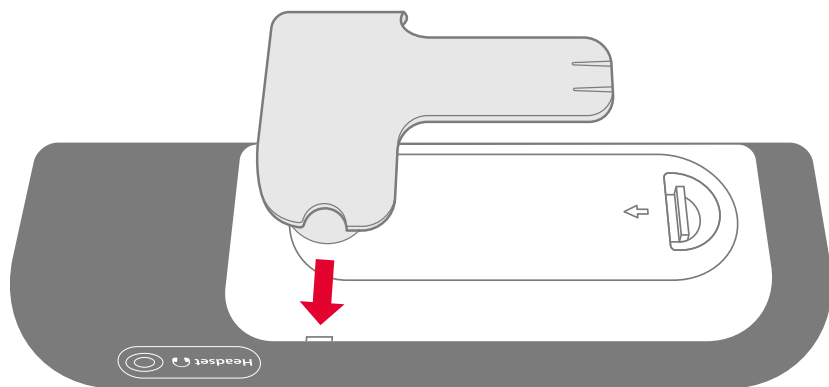
Maxi will automatically reconnect to your TV streamer as soon as it's within range, so you can bring Maxi when you visit family and friends without hassle.

Battery and charging

Maxi is equipped with a rechargeable battery that lasts for up to 70 hours. When the battery indicator (8) starts to blink in red, it's time to charge it, see **Charge Maxi**.

Attaching the clip

To attach the clip, place the clip against the back cover and press firmly until it locks. To remove it, pull the round handles out from the sides.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

⚠ Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Protect cables from any potential source of damage.
- This device has a very powerful amplifier and the sound level can cause discomfort or, in serious cases, endanger hearing if the product is used carelessly. Always turn the volume down to a low level before putting on the headphones!
- Protect the device from shocks during storage and transport.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm, 12" to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- The power adapter must not be exposed to water drops or splashes and objects containing liquids, such as vases must not be placed on the unit. To disconnect the power fully, the adapter plug must be disconnected from the mains socket. The adapter plug must not be blocked and must be easy to access for the intended use. Rating information is shown on the rear.
- Do not block any of the ventilation openings. Install the device in accordance with instructions for use.
- Do not defeat the safety purpose of the polarized or grounding - type plug. A polarized plug has two blades with one wider than the other. A grounding type plug has two blades and a third grounding prong. The wide blade or the third prong are provided for your safety. When the provided plug does not fit into your outlet, consult an electrician for replacement of the obsolete outlet.
- Protect the power cord from being walked on or pinched particularly at plugs, convenience receptacles, and the point where they exit from the unit.
- Unplug this apparatus during lightning storms or when unused for long periods of time.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!
- Only use power adapters and accessories specified in this booklet.

ⓘ Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not drop your device. Dropping onto a hard surface can damage it.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- Rechargeable batteries should be charged fully before using the device for the first

time. Only charge batteries in a temperature between 0°– 35° C, 32°– 95° F.

- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- Do not use the device in areas where electronic equipment is prohibited.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship.

This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Bluetooth® certified

This device includes a Bluetooth module which has passed the BQB certificate. The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by Bellman & Symfon is under licenses.

Product disposal and recycling information

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

The product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

Most problems with Maxi Pro can be solved quickly by following the advice below.

If	Try this
I can't get the units to pair via Bluetooth.	Make sure that your Maxi is in microphone mode by pressing the microphone selector (4). Try pairing the units again by following the steps in the section Pairing.
Nothing happens when I press the Maxi On/Off button.	The batteries are depleted. Charge Maxi, see Charge Maxi.
The Maxi battery indicator is blinking in red.	The battery level is low. Charge Maxi, see Charge Maxi.
I hear no sound in my headphones, even though Maxi is turned on.	- First, make sure the volume is at the required level. Then, check that the headphone plug is properly connected to the Maxi headphone jack (12). - Increase the volume by pressing the + on the Maxi volume controls (6). - Connect the headphones to another device, like a smart phone, and check if you hear any sound. If not, the headphones may need to be replaced. - If Bluetooth mode is selected, make sure that the TV streamer or mobile phone is within range, paired and streaming a sound.
I don't hear the mobile phone in my headphones.	Make sure Bluetooth is activated on your mobile phone and that it is paired with your Maxi, see Pairing in the Using Maxi with your mobile phone section.
I don't hear the sound of the TV in my headphones.	Make sure the TV streamer is within range and paired with your Maxi, see Pairing in the Using Maxi for TV-listening section.
The sound from the TV or mobile is too low, even though the Maxi volume is at the highest level.	Try to increase the volume on your TV or mobile phone instead.
I hear a high-pitched noise in my headphones.	- Lower the volume or increase the distance between Maxi and the headphones. - Direct the Maxi microphone (9) away from you.
Maxi seems unresponsive/frozen.	- Press and hold the On/Off button (3) for ten seconds until Maxi turns off. - Press the On/Off button for about one second to start Maxi again.

Technical specifications

Dimensions and weight

Size: 140 × 27 × 45 mm, 5.5 × 1.1 × 1.8"
Weight: 127g, 4.3 oz.

Frequency and coverage

Bluetooth frequency: 2400 – 2483.5 MHz
Coverage: Up to 25 m, 27.5 yd, free field.
Connectivity: Qualcomm® aptX™ LL

Power supply

Input: 100 – 240V~, Output: DC 5V, 1.0A
Battery power: 2 × 3.7V 750mAh Li-on
Operating time: Up to 70 h
Power consumption: 18 mA

Connections

Headphones input: 3.5 mm mini-stereo jack
Charger input: 2.5 mm jack

Microphone

Built-in omnidirectional microphone

Frequency range

20Hz – 8kHz (Mic), 20Hz – 20kHz (Bluetooth)

Output level with BE9229

135dB @ 1kHz (SPL90) , 136dB (HF Ave. SPL90)
Full-on gain: 49dB @ 1kHz,
Ref. test gain: 49dB @ 1kHz (SPL70)

Output level with BE9233

122dB @ 1kHz (SPL90), 127dB (HF Ave. SPL90)
Full-on gain: 36dB @ 1kHz
Ref. test gain: 36dB @ 1kHz (SPL70)

Mic SNR

53dB @ 1kHz 1% THD (electrical)
Distortion: 0.2% THD @ 1kHz – 6dB @ 1% THD

Bluetooth SNR

85dB @ 1kHz 1% THD, streamer

Output power

Max 82mW @ 1kHz / 16Ω (THD=1%)

Operation

Temp: 0° – 35 ° C, 32° – 95° F
Humidity: 15% to 90%, non-condensing

In the box

- BE2021 Maxi Pro Personal Amplifier
- BE9233 On-ear headset, black
- BE9135 Lanyard
- BE9254 Charge cable USB, white
- Power adapter, white
- Power adapter plug, white

Accessories

- BE9229 In-ear Headset, black
- BE9159 Neck Loop, long cable, black
- BE9161 Neck Loop, short cable, black
- BE2022 Maxi Pro TV Streamer
- BE9147 Travel case

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FCC compliance statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Users are not permitted to make changes or modify the device in any way. Changes or modifications not expressly approved by Bellman & Symfon Group AB will void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains license-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

This device may not cause interference.

This device must accept any interference, including interference that may cause undesired operation of the device.

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All rights reserved.
Bellman & Symfon North America Inc.
Bellman & Symfon Asia Ltd.
TM and © 2025 Bellman & Symfon AB.