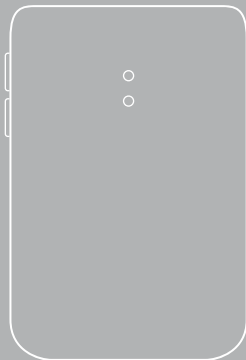
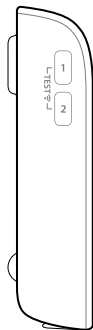
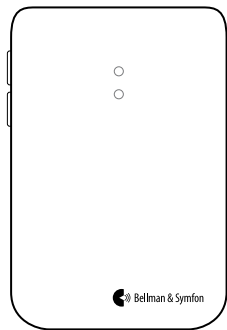
BE1431
Visit Telephone Transmitter

Visit Telephone Transmitter

- | | |
|----|-------------------------|
| 4 | User manual |
| 16 | Manual del usuario |
| 28 | Manuel d'utilisation |
| 40 | 用户手册 |
| 52 | 用戶手冊 |
| 64 | ユーザーマニュアル |
| 76 | Appendix (English only) |

EN

ES

FR

CN_{SI}

CN_{TM}

JP

App

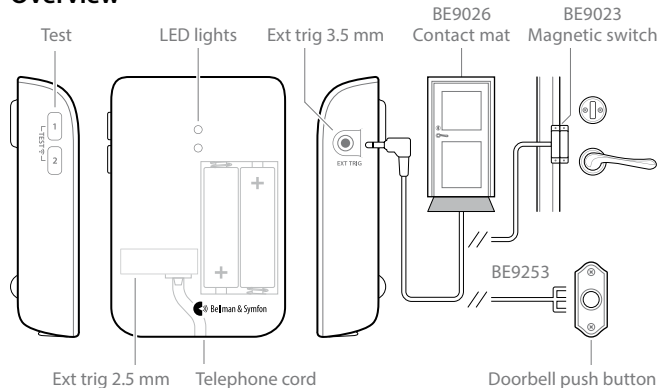
Read this first

This booklet guides you on how to use and maintain your new Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

Intended user group The intended user group are people of all ages.


Principle of operation The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview


Installation

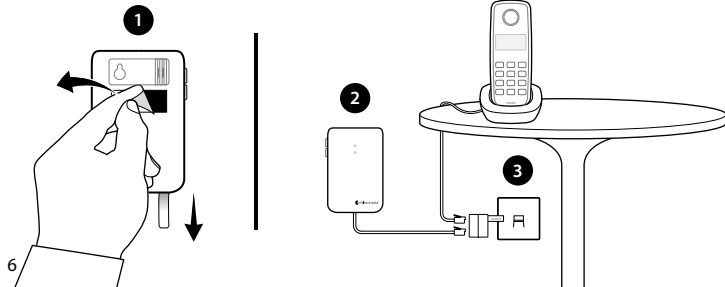- 1 Remove the battery pull tab to start the unit. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the transmitter on the wall. You can also use the supplied screw and plug.
- 3 Connect the telephone splitter between the telephone plugs and the socket.

Testing

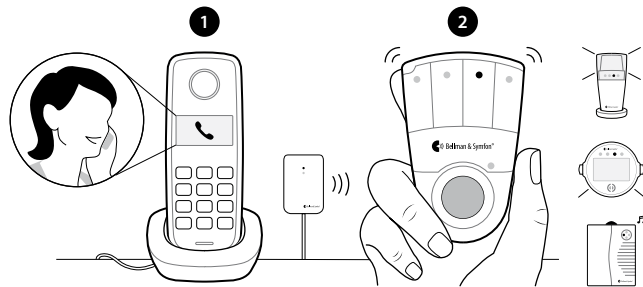- 1 Use your mobile phone or ask a friend to call your landline telephone.
- 2 The yellow Visit LED on the receiver lights up and it starts to sound, flash or vibrate, depending on the receiver. If it doesn't react, see **Troubleshooting**.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.

- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.

- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If

The LEDs blink in orange every minute

The transmitter LED lights up in green but the receiver doesn't respond

The transmitter LED doesn't light up when the phone rings or when an accessory is triggered

The transmitter LED doesn't light up when I press the test buttons

The receiver is activated for no apparent reason

Try this

- The batteries are depleted. Replace them with 1.5 V AA (LR6) lithium or alkaline batteries.

- Check the batteries and connections on all Visit units.
- Move the receiver closer to the transmitter to make sure it's within radio range.
- Check that the units are set to the same radio key, see **Changing the radio key** in the Appendix (English only).

- Press the test buttons. If the LED lights up in green, check all connections. If the LED doesn't light up, replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.

- Replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries. If the LED still doesn't light up, contact your retailer for service information.

- There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see **Changing the radio key** in the Appendix (English only).

Technical specifications

Dimensions	100×65×27 mm, 4.0×2.6×1.1"
Weight	134 g, 4.7 oz. including battery and telephone cord
Power and battery	Batteries: 2×1.5 V AA (LR6) lithium or alkaline Power consumption: Active < 70 mA, Idle position < 15 µA Operating time: Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via test buttons, RJ11 input to analogue telephone network (26-120V RMS, 15-100Hz), 2.5 mm ext trig, 3.5 mm ext trig
Radio frequency	433.92 MHz / 314.90 MHz
Coverage 433 MHz	70 – 120 m, 230 – 394 ft, depending on the building's characteristics
Coverage 315 MHz	30 – 80 m, 98 – 262 ft, depending on the building's characteristics
Operation	Temp: 0° – 35° C, 32° – 95° F, Humidity: 15% to 90%, non-condensing
In the box	- BE1431 Visit Telephone Transmitter 1.5V AA alkaline batteries - Velcro, screws and plugs
Accessories	- BE9026 Contact mat - BE9023 Magnetic switch - BE9253 External trig cable

US: 1PNTX01BBETXTE2

Consumer information: a) This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the bottom of this equipment is a label that contains, among other information, a product identifier in the format US:AAAEQ##TXXXX. If requested, this number must be provided to the telephone company. b) An applicable certification jacks Universal Service Order Codes (USOC) for the equipment is provided (i.e., RJ11C) in the packaging with each piece of approved terminal equipment. c) A plug and jack used to connect this equipment to the premises wiring and telephone network must comply with the applicable FCC Part 68 rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details. d) The REN is used to determine the number of devices that may be connected to a telephone line. Excessive RENs on a telephone line may result in the devices not ringing in response to an incoming call. In most but not all areas, the sum of RENs should not exceed five (5.0). To be certain of the number of devices that may be connected to a line, as determined by the total RENs, contact the local telephone company. [For products approved after July 23, 2001, the REN for this product is part of the product identifier that has the format US:AAAEQ##TXXXX. The digits represented by ## are the REN without a decimal point (e.g., 03 is a REN of 0.3). For earlier products, the REN is separately shown on the label.] e) If this equipment BE1431 causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary. f) The telephone company may make changes in its facilities, equipment, operations or procedures that could affect the operation of the equipment. If this happens the telephone company will provide advance notice in order for you to make necessary modifications to maintain uninterrupted service. g) Should you experience trouble with this equipment, please contact Bellman & Symfon (833) 235-5626 for repair or warranty information. If the equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved. h) Please follow instructions for repairing if any (e.g. battery replacement section); otherwise do not alternate or repair any parts of device except specified. i) Connection to party line service is subject to state tariffs. Contact the state public utility commission, public service commission or corporation commission for information. j) NOTICE: If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this BE1431 does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

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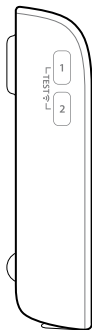
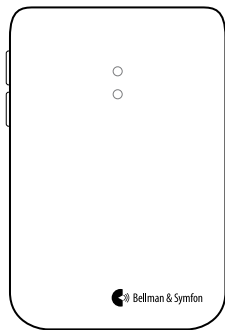
Appendix (EN)

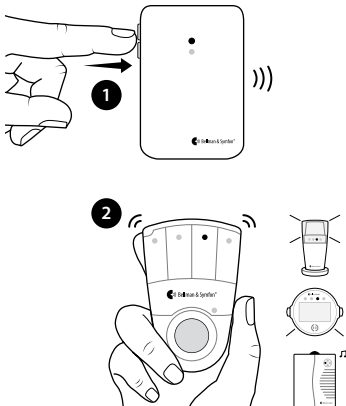
- 78 Testing the signal
- 80 Default signal pattern
- 80 Changing the signal pattern
- 89 Trigger input
- 90 Changing the radio key

Testing the signal

Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The yellow Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

Please note: If you have changed the signal pattern, it will react in accordance with the signal pattern tables on the following pages.


Testing the signal

Using a mobile phone

- 1 Use for instance a mobile phone to call the landline telephone. The top LED on the transmitter lights up in green to show that an incoming call is detected.
- 2 The yellow Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

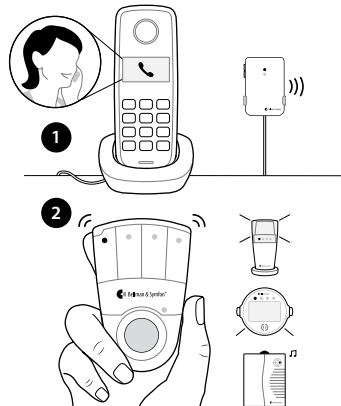Please note: If you have changed the signal pattern, it will react in accordance with the signal pattern tables on the following pages.

Default signal pattern

When the telephone transmitter is activated by an incoming call or a triggered accessory, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in yellow and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

Transmitter | Visit receiver

Source	LED	Sound 315MHz, JPN	Sound 433MHz	Vibration	Flash
Landline phone	Yellow	1 × ring signal, low	1 × door chime, high	Yes	Yes
Mobile phone s.	Yellow	2 × ring signal, high	1 × ring signal, low	Yes	Yes
Accessory	Green	1 × door chime, low	1 × door chime, low	Yes	Yes

Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

Transmitter

Switch

Source

Landline/test button
Mobile phone sensor
Other accessory

Receiver signal pattern

LED

Yellow light
Yellow blinks
Green light

Sound

1 × ring signal, low
2 × ring signal, high
1 × door chime, low

Vibration

Medium
Medium
Slow

Flash

315 MHz

■□□□ Yes
■□□□ Yes
■□□□ Yes

■□□□ Yes
■□□□ Yes
■□□□ Yes

■□□□ Yes
■□□□ Yes
■□□□ Yes

■□□□ Yes
■□□□ Yes
■□□□ Yes

Transmitter Receiver signal pattern 315 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	2×yellow blinks	2×ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1×ring signal, low	Medium	■□■□ Yes
	Other accessory	3×orange blinks	Baby melody	Fast	■□■□■□ Yes
	Landline/test button	2×yellow blinks	2×ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Orange blinks	Baby melody	Fast	■□■□■□ Yes
	Other accessory	Green blinks	2×door chime, high	Slow	■□□□ Yes
	Landline/test button	Orange blinks	Baby melody	Fast	■□■□■□ Yes
	Mobile phone sensor	3×yellow blinks	1×ring signal, high	Medium	■□■□ Yes
	Other accessory	2×green blinks	2×door chime, low	Slow	■□□□ Yes
	Landline/test button	3×yellow blinks	1×ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	2×yellow blinks	2×ring signal, low	Medium	■□■□ Yes
	Other accessory	Green blinks	2×door chime, high	Slow	■□□□ Yes

Transmitter Receiver signal pattern 315 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow blinks	2×ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1×ring signal, low	Medium	■□■□ Yes
	Other accessory	Green blinks	1×door chime, low	Slow	■□□□ Yes
	Landline/test button	Yellow blinks	2×ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1×ring signal, low	Medium	■□■□ Yes
	Other accessory	2×green blinks	2×door chime, low	Slow	■□□□ Yes
	Landline/test button	Yellow blinks	2×ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1×ring signal, low	Medium	■□■□ Yes
	Other accessory	3×yellow blinks	1×ring signal, high	Medium	■□■□ Yes
	Landline/test button	Yellow blinks	2×ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1×ring signal, low	Medium	■□■□ Yes
	Other accessory	2×orange blinks	Baby melody	Fast	■□■□■□ Yes

Transmitter

Receiver signal pattern

315 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	3 × yellow blinks	1 × ring signal, high	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	3 × orange blinks	Baby melody	Fast	■□■□■□ Yes
	Landline/test button	Orange blinks	Baby melody	Fast	■□■□■□ Yes
	Mobile phone sensor	3 × yellow blinks	1 × ring signal, high	Medium	■□■□ Yes
	Other accessory	Green blinks	2 × door chime, high	Slow	■□□□ Yes
	Landline/test button	Orange/Red blink	VMA	Long	■□□□ Yes
	Mobile phone sensor	Orange/Red blink	VMA	Long	■□□□ Yes
	Other accessory	Orange/Red blink	VMA	Long	■□□□ Yes
	Landline/test button	Red blinks	Fire alert	Long	■□□□ Yes
	Mobile phone sensor	Red blinks	Fire alert	Long	■□□□ Yes
	Other accessory	Red blinks	Fire alert	Long	■□□□ Yes

Transmitter

Receiver signal pattern

433 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	Green light	1 × door chime, low	Slow	■□□□ Yes
	Landline/test button	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	Green light	1 × ring signal, high	Medium	■□■□ Yes
	Landline/test button	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Landline/test button	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	Orange light	Baby melody	Slow	■□□□ Yes

App

Transmitter

Receiver signal pattern

433 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Other accessory	Orange light	2 × ring signal, low	Medium	■□■□ Yes
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Orange light	Baby melody	Slow	■□□□ Yes
	Other accessory	Green light	1 × ring signal, low	Slow	■□□□ Yes
	Landline/test button	Orange light	Baby melody	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Green light	1 × ring signal, high	Medium	■□■□ Yes
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Orange light	Baby melody	Slow	■□□□ Yes

Transmitter

Receiver signal pattern

433 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Green light	1 × door chime, low	Slow	■□□□ Yes
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Green light	1 × ring signal, high	Medium	■□■□ Yes
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Orange light	Baby melody	Slow	■□□□ Yes

App

Transmitter


Receiver signal pattern

433 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow light	1 × ring signal, low	Medium	■□■□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Orange light	Baby melody	Slow	■□□□ Yes
	Landline/test button	Orange light	Baby melody	Slow	■□□□ Yes
	Mobile phone sensor	Yellow light	1 × door chime, high	Slow	■□□□ Yes
	Other accessory	Green light	1 × door chime, low	Slow	■□□□ Yes
	Landline/test button	Orange/Red blink	VMA	Continuous	Yes
	Mobile phone sensor	Orange/Red blink	VMA	Continuous	Yes
	Other accessory	Orange/Red blink	VMA	Continuous	Yes
	Landline/test button	Red light	Fire alert	Continuous	Yes
	Mobile phone sensor	Red light	Fire alert	Continuous	Yes
	Other accessory	Red light	Fire alert	Continuous	Yes

Trigger input

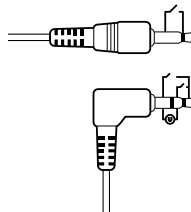You can use both stereo and mono plugs. The 3.5 mm input is located on the top right side and the 2.5 mm input under the lid.

Mono

Trig is made by closing the two connectors

Stereo

Trig is made by:

- 1 Closing the outer and middle connector or closing the outer and inner connector
- 2 Applying 2-30VDC or 3-24VAC 5-150Hz between the middle and inner connector

Please note: When you use the 2.5 mm ext trig, the event must last for more than 2 seconds.

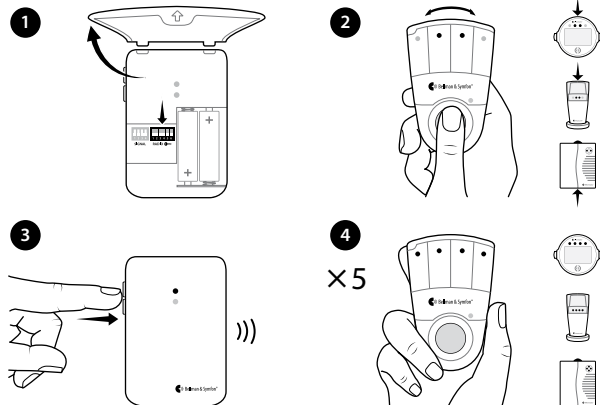
Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink alternately to show that the radio key has been changed. It then returns to normal mode. If you have a BE1033 Portable Receiver, the power LED blinks in amber to confirm.

Please note: All Visit units must be set to the same radio key.

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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FCC ID: WMSBETXE2 **IC:** 6693A-BETXE2

Product disposal and recycling information

According to local laws and regulations your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

This product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

Bellman & Symfon North America Inc.
PO Box 1388, Morrisville,
North Carolina 27560, USA
Phone: (910) 239-5796
Customer support: (833) BELLMAN
or (833) 235-5626

Bellman & Symfon Asia Ltd.
1401 & 1408
Laifung International Centre
33 Jie Fang Road South
Guangzhou, China
Phone: +86 20 8318 2950

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1431_098MAN1.0

Date of issue: 2025-01-13

TM and © 2025 Bellman & Symfon AB.

All rights reserved.