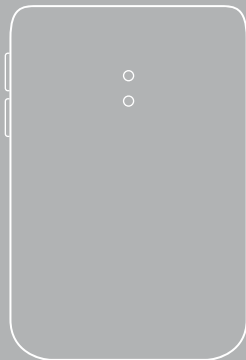
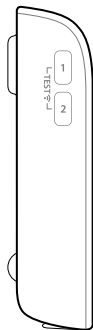
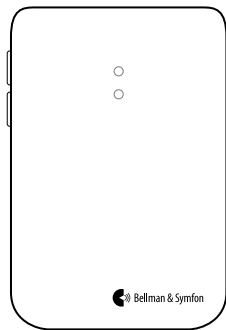
BE1411
Visit Door Transmitter

Visit Door Transmitter

4	User manual
20	Manual del usuario
36	Mode d'emploi
52	用戶手冊
68	用戶手冊
84	ユーザーマニュアル
100	Appendix (English only)

EN

ES

FR

CN_{SI}

CN_{TM}

JP

App

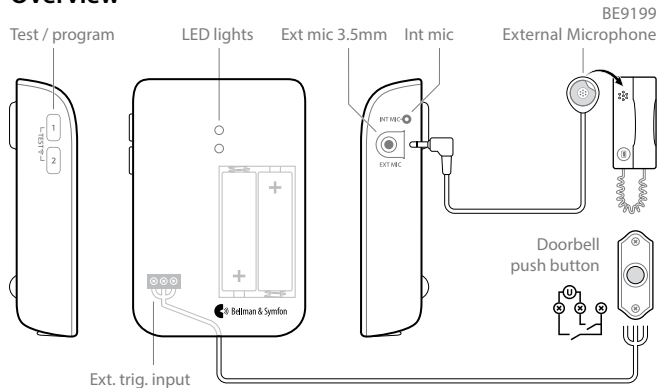
Read this first

This booklet guides you on how to use and maintain your new Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

Intended user group The intended user group are people of all ages.


Principle of operation The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview


Basic installation – single sound source

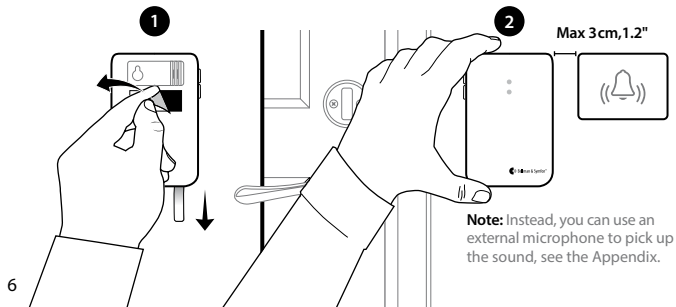- 1 Remove the battery pull tab to start the unit. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. You can also use the supplied screw and plug.

Installation – intercom and electronic doorbell

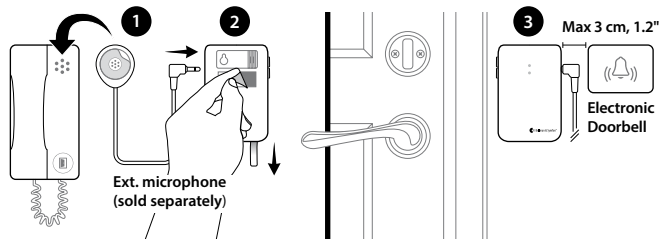
- 1 Remove the protective film from the external microphone (sold separately) and attach it to the intercom's speaker. Connect it to the transmitter.
- 2 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 3 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. Note that the intercom and doorbell must be at least 25 cm, 10" apart to avoid sound interference.

Installation – intercom and mechanical doorbell

If you have an intercom and a mechanical doorbell, you may need two door transmitters, i.e. one for each sound source.

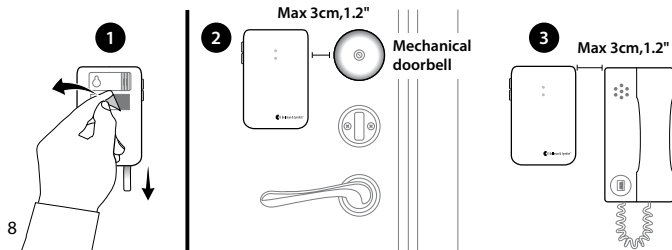
- 1 Pull the battery tab to start each door transmitter. Clean the wall surface with the wet wipe and remove the protective film from the Velcro.
- 2 Mount one of the door transmitters to the left of the doorbell and the other to the left of the intercom's speaker, as close to the sound sources as possible.

Optimizing the doorbell detection


The next step is to teach the transmitter to recognize your doorbell.

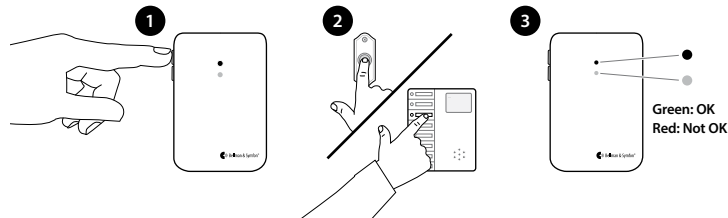- 1 Press and hold button 1 until the top LED start to blink. Release it to start the recording.
- 2 **For door buzzers:** Press and hold the doorbell for at least 8 seconds.
For door chimes: Press the doorbell at least 8 times.
- 3 Press button 1 on the transmitter to stop the recording. If the LED lights up in red, you need to repeat steps 1 and 2. Following the same steps, you can use button 2 to record the sound of your intercom.

Testing the signal

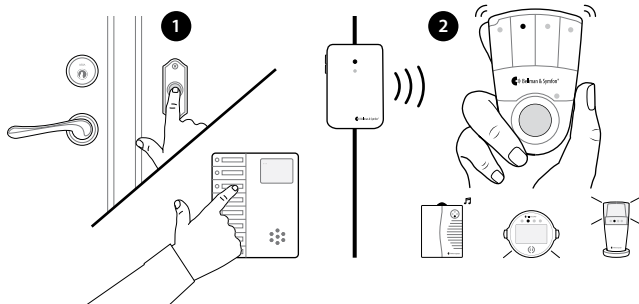- 1 Ring the doorbell or intercom.
- 2 The LED on the Visit receiver lights up in green and the receiver starts to sound, flash or vibrate, depending on the model. If it doesn't react, see **Troubleshooting**.


Sound Level Mode (The transmitter only reacts to dB levels)

- 1 **Prepare the device**
 - Remove the front cover and one of the batteries.
 - Wait for 10 seconds.
 - Hold down test button1 and 2 simultaneously.
 - While still holding down the buttons, refit the battery.
- 2 **Activate Sound Level mode**
 - Keep holding down the test buttons until the green LEDs light up.
 - Release the buttons
 - The unit is now in Sound Level Mode.
- 3 **Choose a sensitivity**
 - Use the test buttons to toggle between the sensitivity thresholds. Red: 65dB, Amber: 75dB and Green: 85dB

To deactivate Sound Level mode

- Repeat the steps in "Prepare the device".
- Release the buttons once the battery is refitted. The LEDs light up in green to confirm.

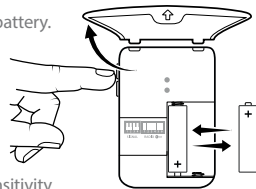Please note: Sound Level Mode may generate unwanted alerts caused by ambient noise.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.

- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If

Try this

The LEDs blink in orange every minute

- Change the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.

The LEDs blink in orange every second

- There are other competing sound sources around the door transmitter. Switch them off or turn them down.
- Disconnect the External microphone to make sure it is not faulty.

The LED on the transmitter lights up when I press the doorbell or intercom - but the receiver is not activated

- Check the transmitter batteries and the receiver batteries and connections.
- Move the receiver closer to the transmitter to make sure it's within radio range.
- Check that the door transmitter and the receiver are set to the same radio key. For more information, see **Changing the the radio key** in the Appendix.

The receiver is activated for no reason

- There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see **Changing the radio key** in the Appendix.

If	Try this
The LED on the transmitter doesn't light up when I ring the doorbell	- Make sure you have mounted the transmitter to the left of the doorbell sound source, no more than 3 cm apart. - If you are using an external microphone, make sure its connected. If you are using an intercom, make sure that the distance between the intercom and transmitter is at least 25 cm or that the internal microphone is turned off. - Try again to teach the transmitter to recognize the sound of your doorbell, see Optimizing the doorbell detection. - If the doorbell signal varies a lot, change to magnetic detection, see Electromagnetic detection in the Appendix.

Technical specifications

Dimensions	100×65×27 mm, 4.0×2.6×1.1"
Weight	100 – 118 g, 3.5 – 4.2 oz. depending on batteries
Power and battery	Battery: 2×1.5 V AA (LR6) lithium or alkaline Power consumption: Active < 70 mA, Idle position < 15 µA Operating time: Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via the test buttons, the internal or external microphone, via electromagnetic detection or the external trigger input.
Max output power	4.7 dBm
Radio frequency	433.92 MHz
Coverage	70 – 120 m, 230 – 394 ft, depending on the building's characteristics
No. of radio keys	64 as standard. Up to 256 with optional software.
Operation	Temp: 15° to 35° C, 59° – 95° F, Humidity: 5% – 90%, non-condensing
Accessories	BE9199 External microphone 2.5 m
In the box	- BE1411 Visit Door Transmitter 1.5V AA alkaline batteries - Velcro, screws and plugs

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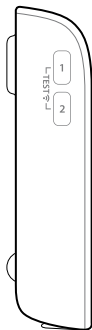
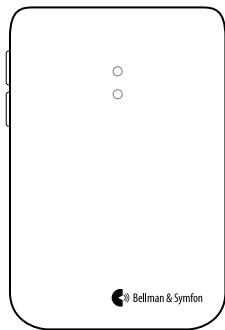
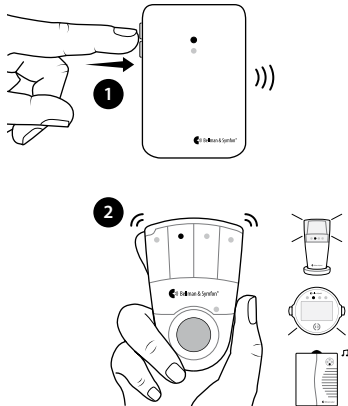
Testing the signal

Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

Please note: If you have changed the signal pattern, it will react in accordance with the signal pattern tables on the following pages.


Testing the signal

Using the doorbell or intercom

- 1 Press the button on the doorbell or intercom. The LED on the transmitter lights up in green to show that the sound is detected.
- 2 The green Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

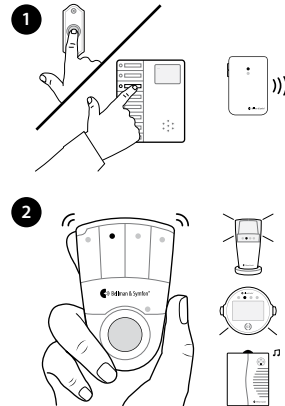Please note: If you have changed the signal pattern, it will react in accordance with the signal pattern tables on the following pages.

Default signal pattern

When the transmitter or a connected accessory is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter determine the signal pattern and the default is as follows:

Transmitter	Receiver signal pattern				
Source	LED	Sound 315MHz, JPN	Sound 433MHz	Vibration	Flash
Internal mic.	Green	1×Door chime, low	1×Door chime, low	Yes	Yes
External mic.	Green	2×Door chime, high	1×Ring signal, high	Yes	Yes
Doorbell con.	Green	1×Door chime, high	1×Door chime, low	Yes	Yes

Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

Transmitter

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1×Door chime, low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green blink	2×Door chime, high	Slow ■□□□	Yes
	Connected doorbell	3×Green blinks	1×Door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	2×Green blinks	2×Door chime low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	3×Green blinks	1×Door chime, high	Slow ■□□□	Yes
	Connected doorbell	Green blinks	2×Door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	3×Green blinks	1×Door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	2×Green blinks	2×Door chime, low	Slow ■□□□	Yes
	Connected doorbell	3×Orange blinks	Baby melody	Fast ■■■■■■	Yes
	Int. mic. / learned signal 1 / test	Green blinks	2×Door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green light	1×Door chime, low	Slow ■□□□	Yes
	Connected doorbell	Orange blinks	Baby melody	Fast ■■■■■■	Yes

Transmitter

Receiver signal pattern

315 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	3 × Green blinks	1 × Door chime, high	Slow	■□□□ Yes
	Connected doorbell	2 × Green blinks	2 × Door chime, low	Slow	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green blinks	2 × Door chime, high	Slow	■□□□ Yes
	Connected doorbell	Yellow blinks	2 × Ring signal, high	Medium	■□■□ Yes
	Int. mic. / learned signal 1 / test	3 × Green blinks	1 × Door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red + Orange blinks	Emergency siren	Long	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 × Door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	2 × Green blinks	2 × Door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red blinks	Fire alarm	Long	■□□□ Yes

Transmitter

Receiver signal pattern

433 MHz

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Long	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 × Ring signal, high	Medium	■□□□ Yes
	Connected doorbell	Green light	1 × Door chime, low	Long	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Ring signal, high	Medium	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, low	Long	■□□□ Yes
	Connected doorbell	Green light	1 × Ring signal, high	Medium	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Long	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 × Ring signal, high	Medium	■□□□ Yes
	Connected doorbell	Orange light	Baby melody	Long	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Ring signal, high	Medium	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, low	Long	■□□□ Yes
	Connected doorbell	Orange light	Baby melody	Long	■□□□ Yes

Transmitter

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Long	Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, high	Medium	Yes
	Connected doorbell	Yellow light	1 × Ring signal, low	Medium	Yes
	Int. mic. / learned signal 1 / test	Orange light	Baby melody	Long	Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, low	Long	Yes
	Connected doorbell	Orange light	Baby melody	Long	Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Long	Yes
	Ext. mic. / learned signal 2	Green light	1 × Ring signal, high	Medium	Yes
	Connected doorbell	Red / Orange blinks	Emergency siren	Continuous	Yes
	Int. mic. / learned signal 1 / test	Green light	1 × Door chime, low	Long	Yes
	Ext. mic. / learned signal 2	Green light	1 × Door chime, high	Medium	Yes
	Connected doorbell	Red blinks	Fire alarm	Continuous	Yes

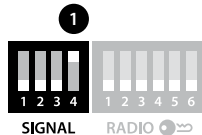
Receiver signal pattern

433 MHz

Using electromagnetic detection

The door transmitter can be set to detect electromagnetic fields emitted by electromechanical doorbells. Here is how it's done:

- **Activating electromagnetic detection**
Move the 4th signal-switch (1) to the Up=On position.
- **Deactivating electromagnetic detection**
Move the 4th signal-switch to the Down=Off position.


Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink alternately to show that the radio key has been changed. It then returns to normal mode. If you have a BE1033 Portable Receiver, the power LED blinks in amber to confirm.

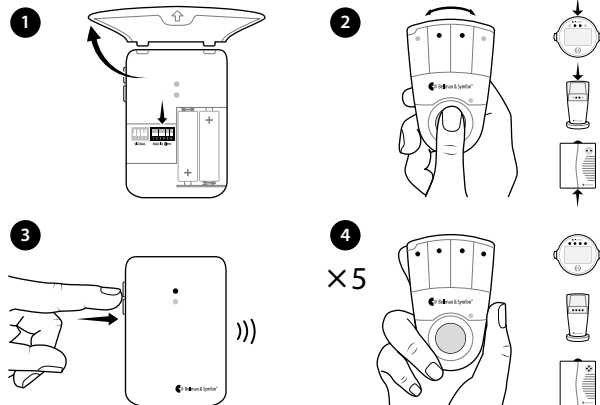Please note: All Visit units must be set to the same radio key.

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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FCC ID: WMSBETXDR **IC:** 6693A-BETX4

Product disposal and recycling information

According to local laws and regulations your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

This product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

Bellman & Symfon North America Inc.
PO Box 1388, Morrisville,
North Carolina 27560, USA
Phone: (910) 239-5796
Customer support: (833) BELLMAN
or (833) 235-5626

Bellman & Symfon Asia Ltd.
1401 & 1408
Laifung International Centre
33 Jie Fang Road South
Guangzhou, China
Phone: +86 20 8318 2950

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1411_152MAN3.0

Date of issue: 2025-02-12

TM and © 2025 Bellman & Symfon AB.

All rights reserved.