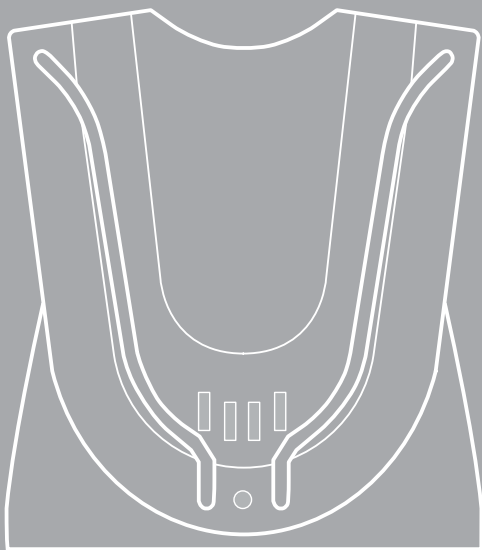
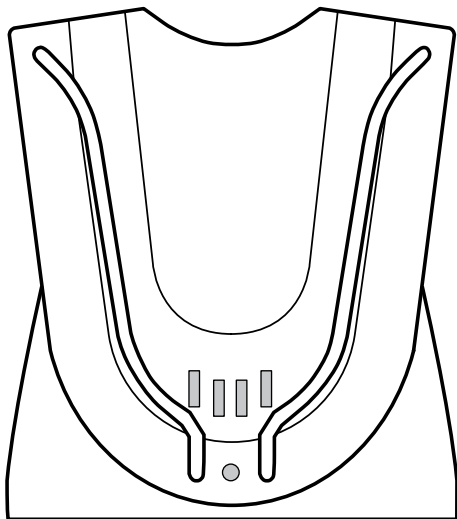
BE1260
Pager Charger

BE1260 Pager Charger

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Read this first

This booklet guides you on how to use and maintain your new Pager Charger. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

Intended user

The intended user is a person with a wish to receive augmented signals.

Intended user group

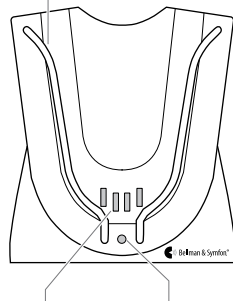
The intended user group consists of people of all ages.

Principle of operation

The Pager Charger is typically placed on the bedside table and used during the night. It can be equipped with up to two BE1270 Bed Shaker accessories that awake the user with vibrations when placed under the pillow or mattress if any of the Visit transmitters are activated.

Overview


Charger brackets

Charging pins

Charging LED

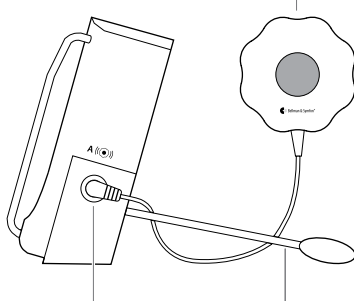BE1270 Bed Shaker


2 × Bed Shaker jacks

Supporting legs

Getting started

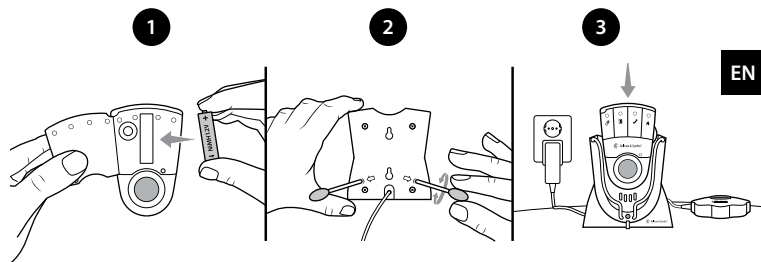- 1 Important!** Replace the Pager's old alkaline battery with the supplied 1.2V NiMH rechargeable battery. Make sure the batteries are inserted correctly.
- Fit the supporting legs to the back of the charger and place it on a level surface. You can also mount it on the wall using the supplied screws and plugs.
- Connect the power supply to the mains outlet and place the pager in the charger. The charging LED is green during charging and goes out when the pager is fully charged. Connect the bed shaker and place under the pillow or mattress.

Testing the connection

Note: Charge the backup batteries for 24 hours before using it with a Bed Shaker.

- To test the radio link, you need the Pager and a Visit transmitter. Press the test button/s on the transmitter (see **Testing the connection** for the relevant transmitter).
- If the Pager is placed in the charger, it lights up a charging LED and the Bed Shaker starts to vibrate. If nothing happens, see **Troubleshooting**.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use.
If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- **Only** use rechargeable NiMH batteries (no less than 600mAh) for the Pager when you are using the Pager Charger. Non-rechargeable batteries will start to leak if the pager is placed in the charger, which might damage the electronics or cause an explosion. This is not covered by warranty.
- Make sure to keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications can be missed if batteries run low of charge.
- This device should not be used as the only means of security in life-threatening situations.
- Use only power adapters and battery types that are specified in this leaflet.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only.
Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.

- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service centre.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0 to 35 °C, 32 to 95 °F.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilisation.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

 Indicates the manufacturer's serial number so that a specific device can be identified.	 Indicates the range of humidity to which the device can be safely exposed.
 This symbol indicates the need for the user to consult the instructions for use.	 Indicates the device manufacturer.
 To identify a place where information may be found, especially in an emergency.	 This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
 Indicates the temperature limits to which the device can be safely exposed.	

Troubleshooting

If	Try this
The pager doesn't charge when it's placed in the charger	- Check that the pager is positioned correctly in the charger. The charging LED will light up in green to show that the pager is being charged. - If the charging LED doesn't light up, the backup batteries may be depleted. Connect the power supply to mains power and charge the backup batteries.
The bed shaker doesn't vibrate when the the pager is activated	- Check that the bed shaker is connected correctly to the charger. - Check that the pager is positioned correctly in the charger. The charging LED will light up in green to show that the pager is being charged. - If the charging LED doesn't light up, the backup batteries may be depleted. Connect the power supply to mains power and charge the backup batteries.

Technical specifications

Dimensions	78×88×43mm, 3.1×3.5×1.7"
Weight	200g, 7oz. incl. batteries and power adapter
Mains power	7.5VDC / 1.5A
Battery power	2×1.2V NiMH rechargeable batteries. The backup batteries must be changed at a service centre.
Pager charging time	Normal charging time: ~ 6h With depleted backup batteries: ~ 24h
Vibrator power	2.0 – 4.0VDC
Charging LED	Green light: The pager is charging No light: The pager is fully charged or the power supply is not connected to mains.
Operation	Temperature: 0 to 35 °C, 32 to 95 °F Humidity: 15% to 90% (non condensing)
Configuration	This BE1260 Pager Charger can be configured with the following Visit devices: - BE1230 Visit Pager receiver - BE1270 Bed Shaker
In the box	- BE1260 Pager Charger with pre-installed backup batteries - External power supply 1 × 1.2V NiMH rechargeable battery intended for the Pager - Supporting legs, screws and plugs

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FCC compliance statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Users are not permitted to make changes or modify the device in any way. Changes or modifications not expressly approved by Bellman & Symfon Group AB will void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains license-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

This device may not cause interference. This device must accept any interference, including interference that may cause undesired operation of the device.

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All rights reserved.
Bellman & Symfon North America Inc.
**Bellman & Symfon Asia Ltd.**
TM and © 2025 Bellman & Symfon AB.