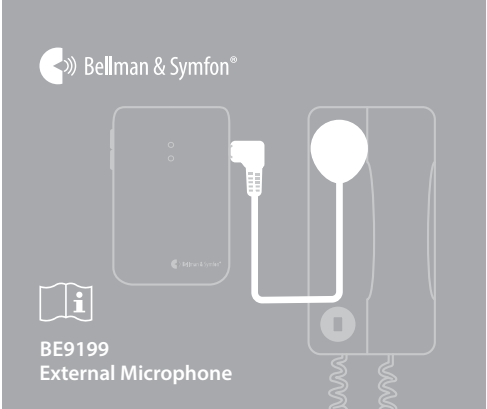
## EN Product information

### Read this first

This booklet guides you on how to use and maintain your new External Microphone. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

### About the BE9199 External Microphone

#### Intended user group

The intended user group consists of people of all ages.

#### Intended user

The intended user is a person with a wish to receive augmented signals.

#### Principle of operation

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

-  The Visit alerting system will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

### General warnings

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

#### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Protect cables from any potential source of damage.

#### Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Do not drop your device. Dropping onto a hard surface can damage it.

### Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

### Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

### Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

### Warranty conditions

Bellman & Symfon guarantees this product for six (6) months from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

### Symbol explanation

|                                                                                     |                                                                                                                                                                                                                                                 |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | Indicates the manufacturer's serial number so that a specific device can be identified.                                                                                                                                                         |
|    | Indicates the device manufacturer.                                                                                                                                                                                                              |
|    | This symbol indicates the need for the user to consult the instructions for use.                                                                                                                                                                |
|   | This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences. |
|  | To identify a place where information may be found, especially in an emergency.                                                                                                                                                                 |
|  | Indicates the temperature limits to which the device can be safely exposed.                                                                                                                                                                     |
|  | Indicates the range of humidity to which the device can be safely exposed.                                                                                                                                                                      |

### Technical specifications

|                        |                                                                               |
|------------------------|-------------------------------------------------------------------------------|
| <b>Weight</b>          | 15 g, 0.5 oz.                                                                 |
| <b>Dimensions</b>      | 33 × 36 × 4 mm, 1,3" × 1,4" × 0,2"                                            |
| <b>Colour</b>          | White                                                                         |
| <b>Cable length</b>    | 2.5 m, 8.2 ft                                                                 |
| <b>Connector</b>       | 3.5 mm mono plug                                                              |
| <b>Microphone type</b> | Piezoelectric                                                                 |
| <b>Operation</b>       | Temperature: 0° to 35° C, 32° to 95° F<br>Humidity: 15% – 90%, non-condensing |
| <b>Configuration</b>   | Intended for BE1411 Visit Door Transmitter                                    |
| <b>In the box</b>      | BE9199 External Microphone                                                    |

-  For detailed product information, see the corresponding user manual.

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All rights reserved.