

Pairing your system



When you turn on the watch for the first time or reset it, either use the watch on-screen guidance or follow the steps below to pair it to your Visit system.

Before you begin, make sure the units are close to each other.



1 Start the bridge

Unpack the bridge and remove the battery pull tab to start it. The top LED will begin to breath in blue.



2 Charge the watch


Unpack the watch and place it in the charger. It will start automatically. Tap **Start** on the watch screen to make it ready for pairing.

Hold down for 3 s



3 Initiate pairing

Hold down the bridge **top** test button until the LEDs start to blink alternately. The bridge will search for your watch and light up the top LED in blue when it has been added.

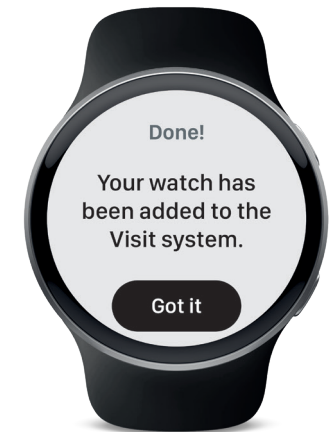

4 Complete the pairing

Skip the guidance steps by tapping **>** until the pairing procedure is completed.

Testing the connection

Make sure to test the connection after you have completed the installation.


Trouble?

If the connection fails, go to the *'Troubleshooting'* section online for guidance.



Troubleshooting

To the watch and phone

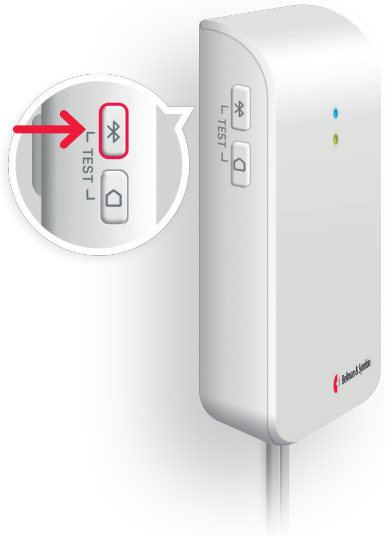

1 On the bridge


Press the **top** test button on the bridge. The top LED lights up in blue to show that a signal is being transmitted.



2 On the watch and phone

A message appears on your watch and mobile phone to confirm that the units are connected to your bridge.

To the Visit system (if applicable)

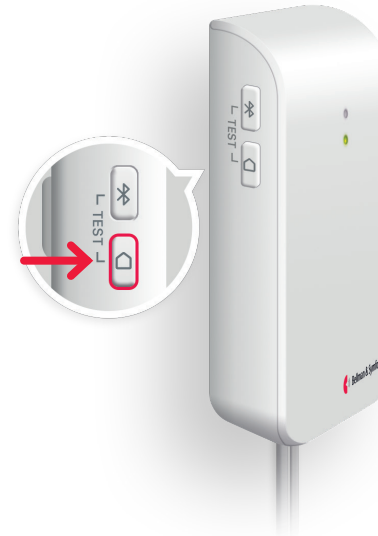

1 On the bridge

Press the **bottom** test button on the bridge. The bottom LED lights up in green to show that a signal is being transmitted.



2 On the Visit receiver

The yellow Visit LED on your receiver lights up and it starts to sound, flash or vibrate depending on the type of receiver.