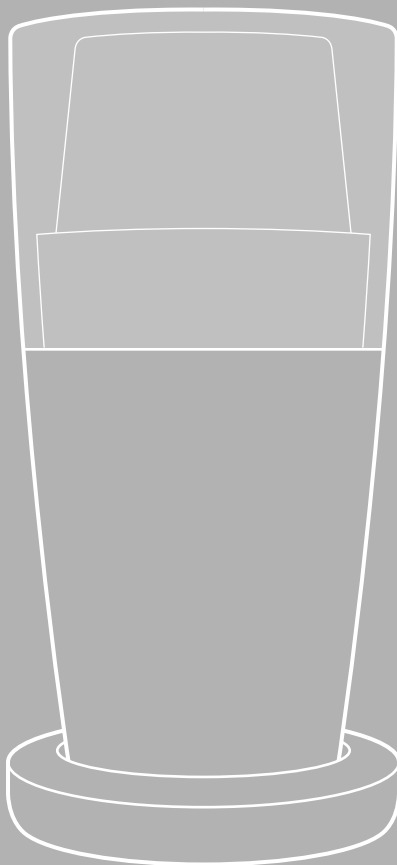
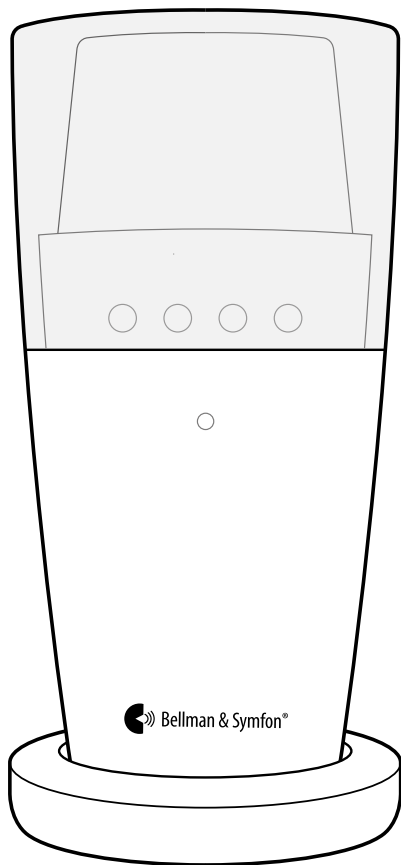
BE1441-868/BE1442-868
Visit Flash Receiver

BE1441-868/BE1442-868 Visit Flash Receiver

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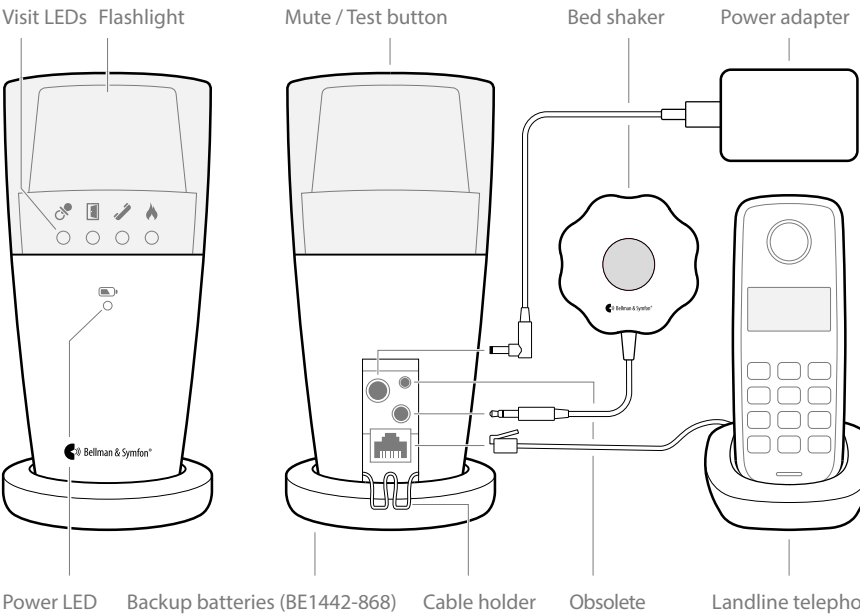
SK

Read this first

This booklet guides you on how to use and maintain your new Visit Flash Receiver. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.


- Intended purpose**
- The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.
- Intended user**
- The intended user is a person with mild to severe hearing loss or deafness.
- Intended user group**
- The intended user group consists of people of all ages with mild to severe hearing loss or deafness who need audio, visual or sensory amplification.
- Principle of operation**
- The Visit system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview

Getting started

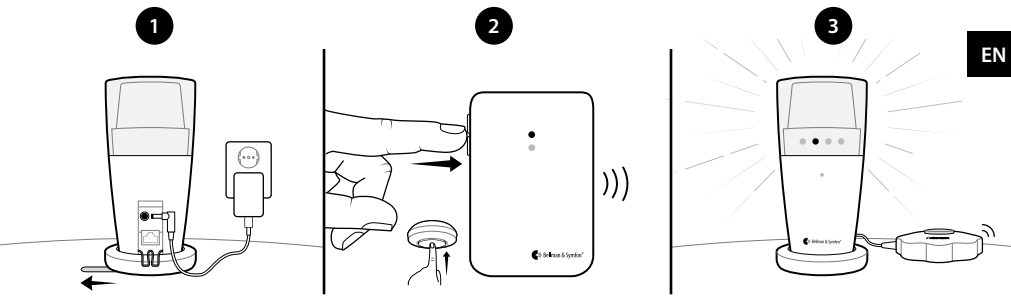- 1 Connect the power supply to the receiver and the mains outlet. Pull the battery tab on the bottom (BE1442-868 only). Place the receiver on a level surface or mount it on the wall using the wall bracket accessory (see separate instructions).
- 2 To test the radio link you need a Visit transmitter. Press the test button/s on the transmitter.
- 3 The receiver lights up a Visit LED and starts to flash. If a bed shaker is connected, it will vibrate. A short press on the mute/test button repeats the last indication. If nothing happens, see **Troubleshooting**.

Default signal pattern

When a transmitter is activated, the flash receiver lights up an LED, flashes and the bed shaker starts to vibrate with a certain pace. This is called signal pattern. The transmitters determine the pattern, and the default is as follows:

Transmitter	Flash Receiver		Bed Shaker
	Visit LED	Flash	
Activated source			Vibration
▪ Door transmitter	Green	Yes	Slow
▪ Push button transmitter	Green	Yes	Slow
▪ Telephone transmitter	Yellow	Yes	Medium
▪ Baby monitor	Orange	Yes	Fast
▪ Smoke alarm	Red	Yes	Long
▪ CO alarm	Orange/red	Yes	Long

Changing the signal pattern

The signal pattern can only be changed on the transmitters. See **Changing the signal pattern** for the relevant transmitter.

Power LED indications

When the flash receiver is connected to mains power, the power LED lights up in green. The BE1442-868 model is also equipped with battery backup and the power LED indicates the following:

Power LED	Status
▪ Green light	The flash receiver is connected to mains power. The backup batteries are detected.
▪ Green blinks	The flash receiver is connected to mains power. No backup batteries are detected.
▪ Red light	The flash receiver is running on battery backup.
▪ Red blinks	The backup batteries are nearly depleted.

Personalizing your receiver

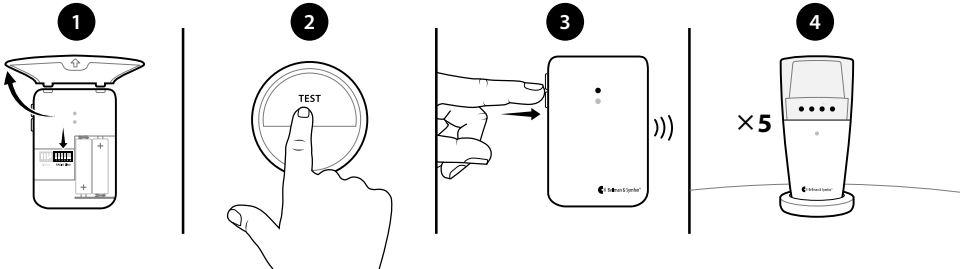
Directing the flash

The flash receiver features a rotating top that makes it easy to direct the light. Point it for example towards a wall if you feel that the flash is too intense. A silicone slip-on top is also available in a variety of colors (art. no. BE9164-BE9167).

Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located on the **transmitters**.


- 1 Open the transmitter cover and move any radio key switch to the up (on position) to change the radio key. See **Changing the radio key** for the relevant transmitter.
- 2 Press and hold the test button located on the top of the flash receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the test button/s on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. Please note that all Visit units must be set to the same radio key.

Testing the connection


A Visit transmitter set to the same radio key as the flash receiver is required to test the system. You will need a bed shaker accessory to test the vibration.

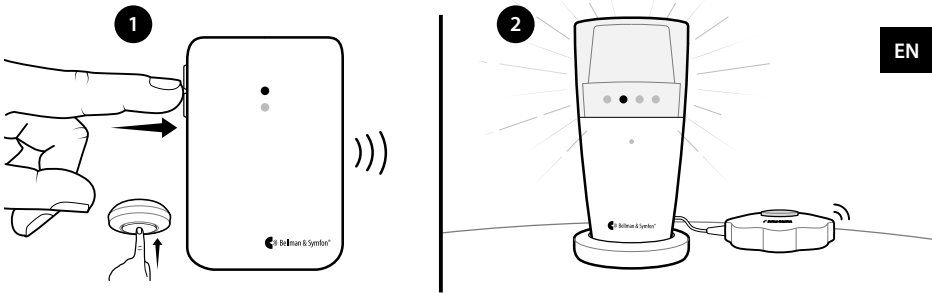- 1 Press the transmitter test button(s).
- 2 The flash receiver will:
 - Start to flash
 - Light up the corresponding Visit LED
 - If a bed shaker accessory is connected, it will start to vibrate.


Testing the Landline telephone

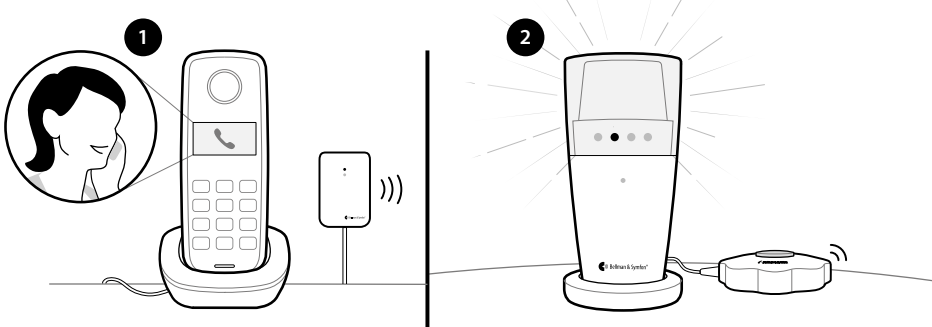- 1 Connect the flash receiver to an analogue telephone socket via the telephone Input. You will need a telephone flex cable accessory and an adapter plug accessory.
- 2 Use for instance your mobile phone to call your Landline telephone. The flash receiver will start to flash and light up the yellow Visit LED. If a bed shaker accessory is connected, it will start to vibrate. To repeat the last indication, press the flash receiver test button.


Battery backup

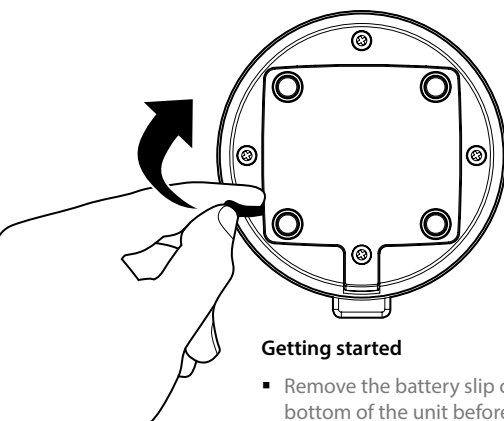BE1442-868

Getting started

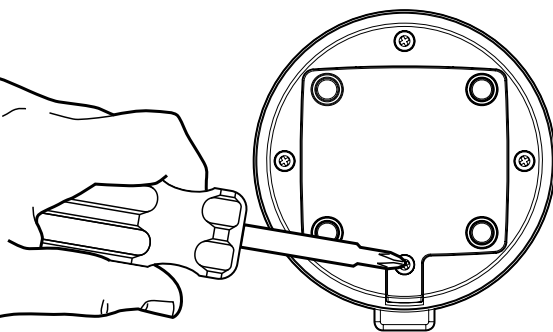- Remove the battery slip on the bottom of the unit before use.
- Please charge the unit for 24 hours to ensure full backup time.


Replacing batteries (Done by the operator only)

- Remove the screw on the bottom of the unit to exchange the batteries.

⚠ Caution

Risk of explosion if battery is replaced by an incorrect type.

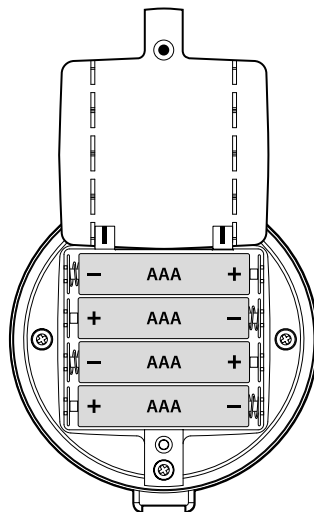Dispose of used batteries according to the instructions in this user manual.

- Use only 4 × 1.2V NiMH 600AAA/900AAA Lexel rechargeable batteries to ensure full backup time.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

⚠ Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications can be missed if batteries run out.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Flashes can trigger epileptic seizures for people with photosensitive epilepsy (PSE).
- Use only power adapters and battery types that are specified in this booklet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

i Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- The programming pin on the back of the flash receiver is for authorized service personnel only. Unauthorized use may damage the device.

EN

- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship.

This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Troubleshooting

If	Try this
The receiver seems to be turned off	▪ Check that the power supply is connected correctly. ▪ Charge the backup batteries for at least 24 hours (BE1442-868 only).
The power LED blinks in red	▪ The backup batteries are nearly depleted and the power supply is disconnected. Connect the power supply and charge the batteries for at least 24 hours.
The power LED blinks in green	▪ The receiver detects no backup batteries. Pull the battery tab, see Getting started.
The receiver does not respond when a transmitter is activated, but works when I use the test button	▪ Check the transmitter batteries and connections. ▪ Move it closer to the transmitter to make sure it's within range. ▪ Check that the receiver is set to the same radio key as the other units in the Visit system, see Changing the radio key.
The receiver is activated for no apparent reason	▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key.
The flash is too bright	▪ Redirect the light by rotating the top or use a silicone slip-on top accessory to dim the light.

Technical specifications

Dimensions	BE1441-868: 70 × 140 mm, 2.7 × 5.5" (Ø × H) BE1442-868: 78 × 155 mm, 3.1 × 6.1" (Ø × H)
Weight	308 g, 10.8 oz. (both models)
Mains power	7.5 V DC / 1500 mA Only use the supplied power adapter. This product is also designed for IT power distribution systems with phase to phase voltage of up to 230V. The outlet shall be installed near the equipment and shall be easily accessible.
Power consumption	Active: 750 mA, idle position: 8 mA
BE1442-868 Battery backup	Battery type: 4 × 1.2 V Lexel 600/900 mAh Operating time: ~ 48 h when fully charged
Connections	▪ 2.5 mm external trigger jack ▪ 3.5 mm bed shaker outlet, vibration power: 2.0 – 4.0 VDC (max. 500 mA) ▪ RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz ▪ Barrel jack for the external power adapter ▪ Programming pin for service purposes. Output: 3.3 V/max. 150mA.
Flash	Built-in ~30 Candela Xenon light Warning! Flashes can cause epileptic seizures
Radio frequency	868.30 MHz
Coverage	50 – 250 m, 55 – 273 yards depending on the characteristics of the building.
Configuration	This receiver can be used with all Visit transmitters.
Operation	Temperature: 0° to 35° C Humidity 15% to 90%, non-condensing
Accessories	▪ BE9075 Wall mount bracket ▪ BE1270-P02 Bed Shaker ▪ BE9105 Telephone Cable
In the box	▪ BE1441-868 Visit Flash Receiver or BE1442-868 Visit Flash Receiver with battery backup ▪ 7.5 V DC / 1500 mA power adapter ▪ 4 × 1.2 V AAA NiMH batteries (BE1442-868 model only)

Symbol explanation

	Indicates the item is a medical device.
	Indicates a carrier that contains unique device identifier information.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Indicates the medical device manufacturer.
	This symbol indicates the need for the user to consult the instructions for use.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	To identify a place where information may be found, especially in an emergency.
	Indicates the temperature limits to which the medical device can be safely exposed.
	Indicates the range of humidity to which the medical device can be safely exposed.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website bellman.com.

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM and © 2024 Bellman & Symfon AB.