
BE1270-P02  
Bed Shaker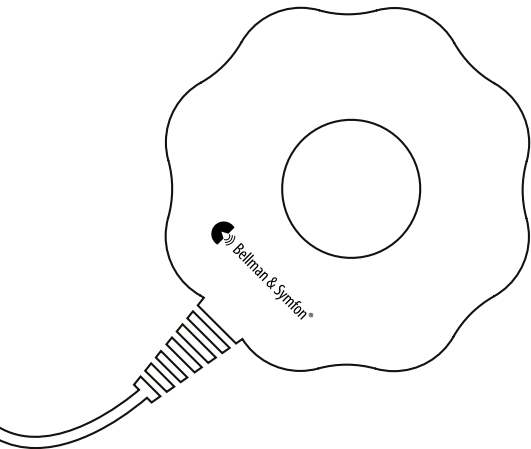

## BE1270-P02 Bed Shaker

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SK

## Read this first

This booklet guides you on how to use and maintain your new Bed Shaker. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

**Intended purpose** The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.

**Intended user** The intended user is a person with mild to severe hearing loss or deafness.

**Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

**Principle of operation** Place the Bed Shaker under your mattress or pillow and plug it into any of the Bellman & Symfon receivers or chargers. If any of the Visit transmitters are activated, the Bed Shaker will wake the user with vibrations.

The Visit alerting system will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.


## Getting started

The Bed Shaker can be used with all the Visit receivers and chargers, as a means of waking up with help of vibrations.

- 1 Connect the Bed Shaker by inserting the connector in the Bed Shaker jack on the receiver or charger. (See below)
- 2 Press the test button on any of your Visit transmitters and check that the Bed Shaker vibrates.

**Note:** When you connect the Bed Shaker to the pager charger, the pager must be inserted in order for it to work.

EN

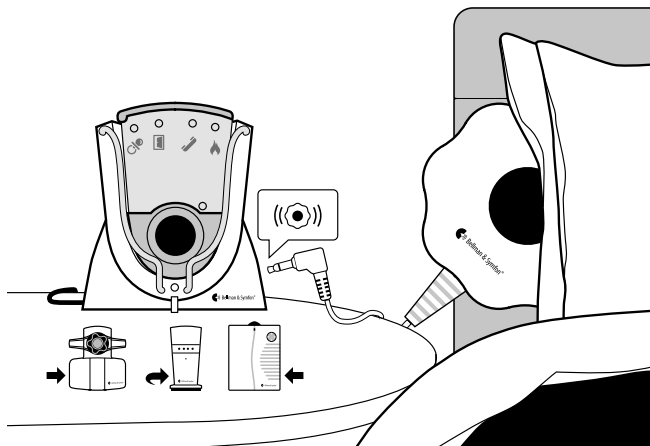## General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Protect cables from any potential source of damage.

### Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.

- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](http://bellman.com) for contact info.
- Do not drop your device. Dropping onto a hard surface can damage it.
- When you connect the bed shaker to the pager charger, the pager must be inserted in order for it to work.

### Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

### Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

### Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.


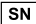
### Warranty conditions

Bellman & Symfon guarantees this product for six (6) months from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with

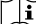territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Symbol explanation

	Indicates the item is a medical device.
	Indicates a carrier that contains unique device identifier information.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Indicates the medical device manufacturer.
	This symbol indicates the need for the user to consult the instructions for use.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	To identify a place where information may be found, especially in an emergency.
	Indicates the temperature limits to which the medical device can be safely exposed.
	Indicates the range of humidity to which the medical device can be safely exposed.

## Technical specifications

<b>Dimensions</b>	88 × 88 × 27 mm, 3.5 × 3.5 × 1.1"
<b>Weight</b>	120 g, 4.2 oz
<b>Operating voltage</b>	2.0 - 4.0 V DC from a Visit receiver or charger
<b>Power consumption</b>	250 - 750 mA
<b>Color</b>	White with red rubber anti-skid protector
<b>Cable length</b>	2 meters
<b>Function</b>	The BE1270-P02 Bed shaker is a vibrator for indoor use that is placed in the bed to attract the attention of the user with vibrations.
<b>Installation</b>	The Bed Shaker is usually placed under a pillow or under the mattress. It is connected to a range of Bellman & Symfon products with vibrator outputs.
<b>Operation</b>	Temp: 0° to 35° C Humidity: 15% to 90%, non-condensing
<b>Configuration</b>	This device can be used with all Visit receivers
<b>Care and cleaning</b>	Maintenance free. Clean with a soft cloth. Do not use cleaners, solvents, ammonia or alcohol.
<b>In the box</b>	■ BE1270-P02 Bed Shaker

EN

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website [bellman.com](http://bellman.com).

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TM and © 2024 Bellman & Symfon AB. All
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB